


Implement a world-class cleaning monitoring program.

A reliable cleaning monitoring system for endoscopes, surgical instruments, and high-touch surfaces in patient care settings.

We all know the dangers that inadequately cleaned surfaces, endoscopes and other devices present — to patients, and to your healthcare facility's reputation.

The 3M™ Clean-Trace™ ATP Monitoring System provides rapid, real-time results and quantitative data and reporting to support your cleaning monitoring program.

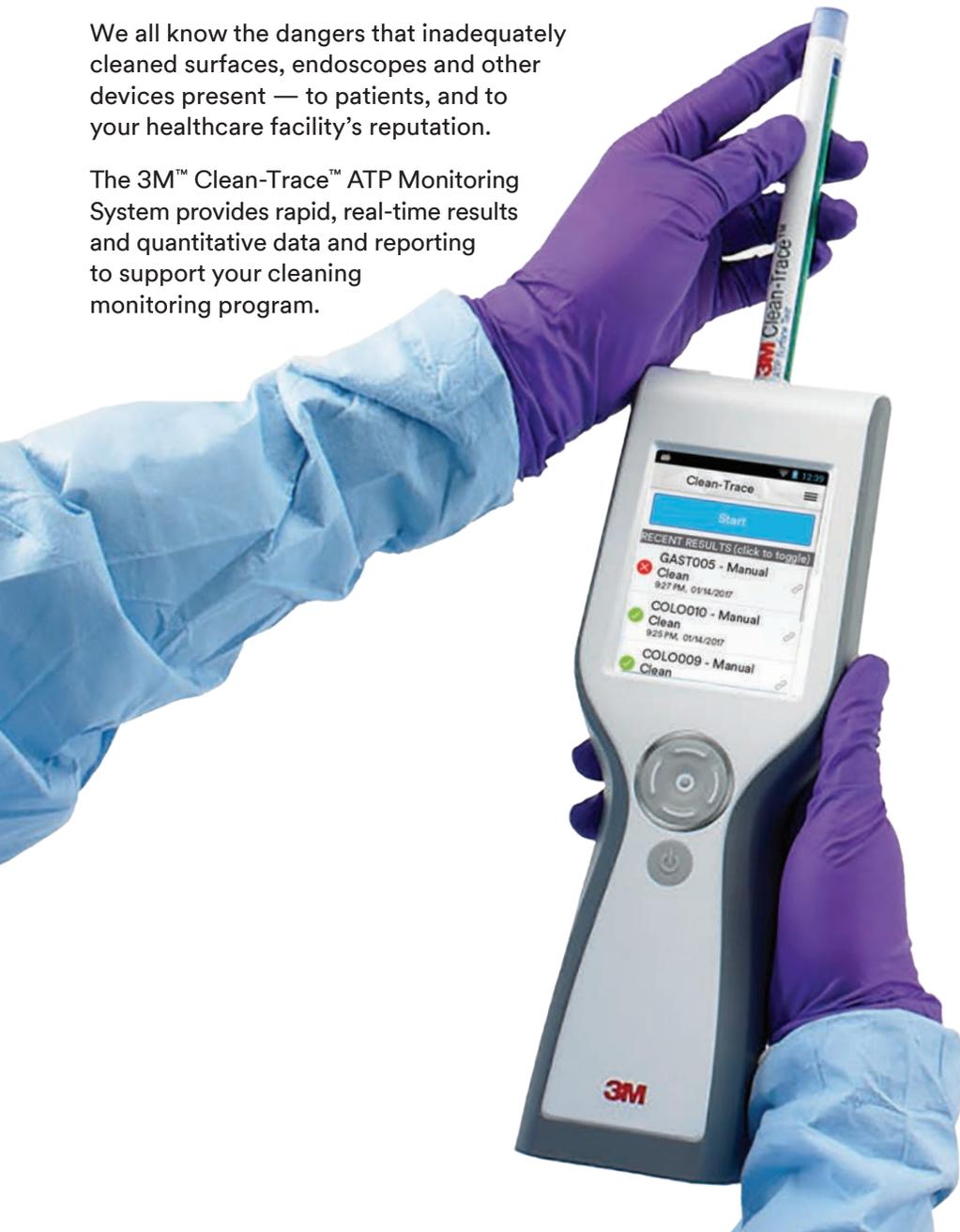

The 3M™ Clean-Trace™ Luminometer LX25. Accurate, consistent and designed to be user friendly.

The Luminometer LX25 was designed based on user feedback with new functionality, a color touchscreen and a modern ergonomic design.

After sampling, insert the 3M™ Clean-Trace™ ATP Surface or Water Test. The Luminometer measures, displays and records the results. Data is transmitted wirelessly to the 3M Quality Control Data Manager (QCDM). Results are accurate, consistent and highly repeatable.

- Provides accurate Pass/Fail results and RLU measurement in ≤ 10 seconds*
- Automatic data syncing to the 3M QCDM when connected to WiFi
- Ergonomic design
- Easy to open test chamber for easy sample insert
- Color touchscreen
- Battery recharge in 3 hours when connected to a power supply

*Pass/fail RLU threshold values are determined by each healthcare facility.

3M™ Quality Control Data Manager (QCDM)

Get a clear picture of what's working and what's not.

The 3M QCDM securely captures and stores data from all test points, then analyzes and translates it into dashboards and reports for easy analysis. You'll see visual data summaries the moment you log in with the new summary dashboards.

This online, hosted service helps keep you on top of your facility's cleaning process.

You can easily create, pull and schedule custom reports within or across your facility or system, including:

- Organization
- RLU Distribution
- Staff
- Test Points
- Rooms/Areas
- Devices



Intuitive dashboard

Provides quick assessment of cleaning monitoring program performance. Helps identify problem areas with the automatic "Watchlist."

Auto-generated reports and data download

Flexibility to choose by report type, recipients, scheduled frequency. Easy download of information.

Be audit ready

Your monitoring data and reports help demonstrate that you're meeting facility-established benchmarks and are compliant with policies and procedures.

3M support

Service, software updates and technical expertise and support come with every purchase.

- Set-up, Implementation and Sampling Guides
- Field based support for monitoring plan development, implementation and training
- 3M Health Care Helpline 1-800-228-3957

Catalog Number	Description	Packaging	Shipping
LX25	3M™ Clean-Trace™ Luminometer	1 each/box	1 box/case
QCDM	3M™ Quality Control Data Manager	—	—
UXC	3M™ Clean-Trace™ Surface Test	10/pouch	10 pouches/case
H2O	3M™ Clean-Trace™ Water Test	10/pouch	10 pouches/case

The 3M™ Clean-Trace™ Luminometer LX25 comes with a one-year limited warranty. A Two-year extended care plan is available for purchase and includes annual calibration and repair. Manufacturer warranty details (see user manual) and extended care plan options for the 3M™ Clean-Trace™ Luminometer can be found on 3M.com/CleanTrace.



3M Company

3M Center, Building 275-4E-1000
St. Paul, MN 55144-1000 USA
800-228-3957
3M.com/Medical

© 2020 3M. All rights reserved. 3M and the other marks shown are marks and/or registered marks. Unauthorized use prohibited. 70-2011-6855-9