**CINTURA POLIVALENTE - CTBL206S (standard) CTBL206E (extra)
BODY BELT - CTBL206S (standard) CTBL206E (extra)**

Descrizione:

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Intended Use:

The Abdominal Support Belt is designed as an alternative to the traditional patient support vest in bed or chair-type structures.

Device intended exclusively for abdominal positioning. Do not apply to any other part of the body. To keep a patient in a sitting position in a chair or bed and prevent possible falls or slips.

Composition:

*Realized with a hypoallergenic and breathable material.
Equipped with wide straps for fastening the patient.
Washable at 40°/60°C.
Internal-external layer: Polyamide 100%
100% Polyurethane foam padding
Straps: Cotton 100%
Ring: Nylon
LATEX FREE*

Instructions for Use:

*Place the belt over the patient's abdomen. The large model incorporates a Velcro fastening system, so the abdominal belt can be fastened to the patient's lower back.
Cross the strips over the lumbar area.
Put the straps over the abdominal area, insert them through the respective side rings and make a safety knot. Take the straps back around to the back.
Tie to the back of the wheelchair or bed.*

Product Characteristics:

It allows more stretch from the bottom and top of the product than from the center. The product retains the shape without the need for uncomfortable buckles. The strong straps give strength and durability to the product. Increases security. Breathable material that can be easily washed. Fresh and comfortable. It incorporates a cushion fresh and breathable. Patient comfort. Extra long to be able to bind to the structure of the bed or wheelchair.

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IN COMPLIANCE WITH:

Directive 93/42/EEC of the Council, Directive 2007/47/EC of the European Parliament and of the Council and Regulation (EU) 2017/745 of the European Parliament and of the Council. Class I, non-sterile (according to Annex IX of Directive 93/42 and Annex VIII of Regulation 745/2017).
 Enterprise certified by ISO 13485 and ISO 9001.
 Medical Devices Manufacturer Operating License No.: 3689-PS.

Precautions and Recommendations:

The use by more than one patient may cause a cross-infection.
 Do not use the product on injured skin.
 Do not use the product if it is dirty or evidently deteriorated.

Storage and Shelf-Life:

Keep away from sunlight. Keep dry.
 Shelf life: 5 years

Code	Size	L (cm)	W (cm)	STRAP L (cm)
CTBL206S REF. 1070100	Standard (medium)	70	17	147
CTBL206E REF. 1070150	Extra (large)	115	17	167

Packaging:

Each unit in a clear plastic bag of polypropylene.
 CTBL206S REF. 1070100:
 12 units in cardboard box.
 CTBL206E REF. 1070150:
 12 units in cardboard box.
 Product for one patient only.

CE certified according to EU regulation 2017/745 on Medical Devices.

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