GARANZIA

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WARRANTY

This bath bench has **2-YEAR** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get your bath bench repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

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I class medical device
according to ECC directive 93/42

Code RS918-919

Thank you for purchasing bath bench **MOPEDIA** by Moretti, which have been manufactured according to the highest quality and safety standards. Powder coated steel tube frame with a base plate to secure it to the wall. All seats can be put in an upright position. Model RS919 has height adjustable legs which are required when extra support is needed.

- **RS918**
- **RS919** h 38 - 52 cm

Assembly Instructions

The mounting plate has mounting holes to secure to the wall. It should be fitted by a competent person and secure to a wall capable of supporting the vertical load and leverage affected upon by the user. RS918 maximum user weight is 110 kg. RS919 maximum user weight is 110 kg.

To adjust leg height (only RS919)

To adjust the height, grasp the horseshoe clip. Slide the leg out to the required position and push the clip firmly back into the hole.

NOTE: set the leg and rail height before securing the mounting plate to the wall.

Cleaning

Mopedia lift-up shower are designed for minimum maintenance and require the occasional wipe over with a soft cloth. Use general purpose neutral detergent and warm water.

Don't use harsh abrasives or bleach based cleaners which may cause damage or discoloration. Periodically check the wall plate fittings and the pivot point of the rail are secure.

Warning

The manufacturer and distributor decline all responsibility for any consequences resulting from an incorrect use of this product.

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the use, adjustment or maintenance of the product, please contact your dealer.
- Worn, damaged or missing components must be replaced.
- Do not store the product outdoors.
- Periodically check the stability of the product and that the screws are properly tightened and the rubber caps ends in good conditions (only for RS919).