

MA0RA303 - ANTI-DECUBITUS KIT WITH MAT AND COMPRESSOR WITH ADJUSTMENT

INTENDED USE

The line of compressor and mattress kits is designed for the prevention and treatment of bedsores that can occur in all those situations that require a long hospital and / or home stay.

GENERAL WARNINGS

- For proper use of the device refer carefully to the following manual
- For proper use of the device always consult your doctor or therapist
- Keep the packaged product away from any heat source as the packaging is made of cardboard
- The useful life of the device is determined by the wear of non-repairable and/or replaceable parts
- Always pay attention to the presence of children
- The user and/or patient shall report any serious incident occurring in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

INSTALLATION

Unpack the device to verify that it has not suffered any damage during shipping. If you find any damage, contact your dealer immediately.

1. Place the inflatable mattress on top of a regular mattress so that the fittings for connecting the air hoses are placed towards the foot of the bed.
2. Hang the compressor at the foot of the bed using the hooks on the back of the compressor, or place it on a flat surface
3. Connect the two connecting hoses to the compressor and mattress connectors
4. Connect the plug to the electrical outlet
5. Turn on the switch located on the right side of the compressor.

NOTE:

1. Check that the voltage of the electrical system is suitable for the compressor
2. The socket can also serve as a switch to disconnect the appliance

ATTENTION!

- The pump can only be applied to the mat recommended by the manufacturer.
- Do not use it for any other purpose. (Applied element: air mat)
- It is advisable to place the mattress with interchangeable elements on top of a mattress with a height of not less than 5 cm.
- Check that there are no bends or crushing obstructing the passage of air.
- Do not smoke on or near the pump.
- Keep the pump away from heated surfaces.
- Risk of explosion if used in the presence of flammable anesthetics.

Some installation tips:

After installation, roll up any cable that is left free to avoid tripping. The device must be placed in a place that is fully accessible to the user or doctors.

BEFORE EACH USE

Check the correct assembly of the device with particular attention to the connection of the pipes between the mattress and the compressor.

CLEANING INSTRUCTIONS

Mattress

Clean the mattress regularly with neutral detergent or alcohol. Hot or steam sterilization is not possible.

Pump

Clean the pump regularly with mild detergent or alcohol. Do not open the pump housing - risk of electrocution. Do not wet or immerse the pump in any liquid


DISPOSAL CONDITIONS

In case of disposal of the device, never use the normal municipal solid waste delivery systems. Instead, it is recommended to dispose of the device through the common municipal ecological islands for the planned recycling operations of the materials used.

Warnings for the correct disposal of the product according to European Directive 2012/19/EU:

At the end of its useful life, the product must not be disposed of together with municipal waste. It can be delivered to the appropriate separate collection centers set up by the municipal administrations, or to retailers that provide this service. Disposing of the product separately avoids possible negative consequences for the environment and health deriving from its inadequate disposal and allows to recover the materials of which it is composed in order to obtain an important saving of energy and resources. To underline the obligation to dispose of electro-medical equipment separately, the product bears the mark of the crossed-out mobile garbage container.



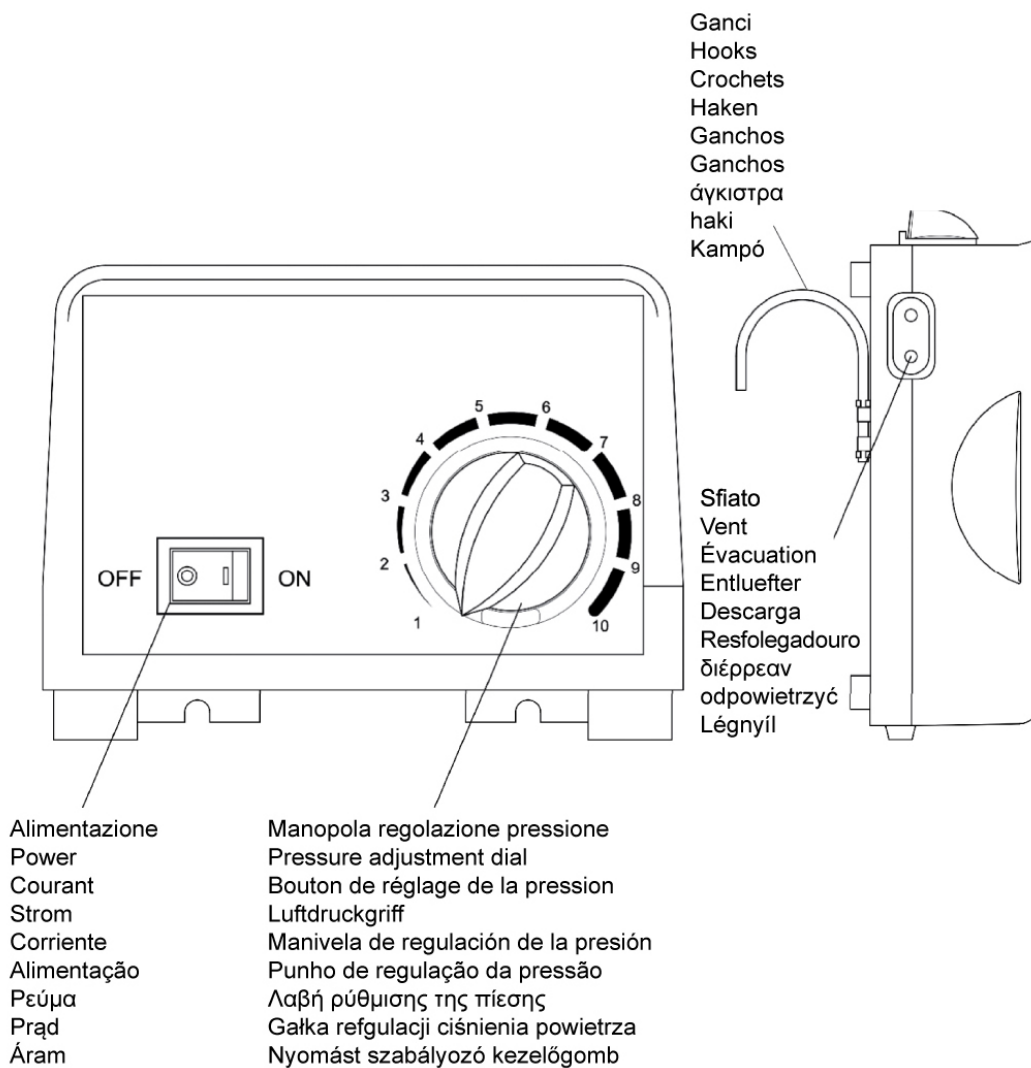

REF

LOT

MA0RA303

See the label below the device

(01)08051882210393(10)xxxxxx

Compressore antidecubito con materasso a bolle



RAM Apparecchi Medicali srl
Via Casaregis 19/25r – 16129 Genova – Italy
+39.010.5761476 - info@anteamed.it

