GIMA

SFIGMOMANOMETRI DALLAS
SPHYGMOMANOMETERS DALLAS - TENSIOMETRES DALLAS
ESFIGMOMANOMETROS DALLAS - BLUTDRUCKMESSGERÄTE
DALLAS - ESFIGMOMANMETROS DALLAS
ΣΦΥΓΜΟΜΑΝΟΜΕΤΡΑ DALLAS

MANUALE D'USO E MANUTENZIONE
USE AND MAINTENANCE BOOK
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN
BETRIEBS UND WARTUNGS ANWEISUNGEN
MANUAL DE USO Y MANTENIMIENTO
MANUAL DE USO E MANUTENÇÃO
Οδηγίες χρήσεως

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ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

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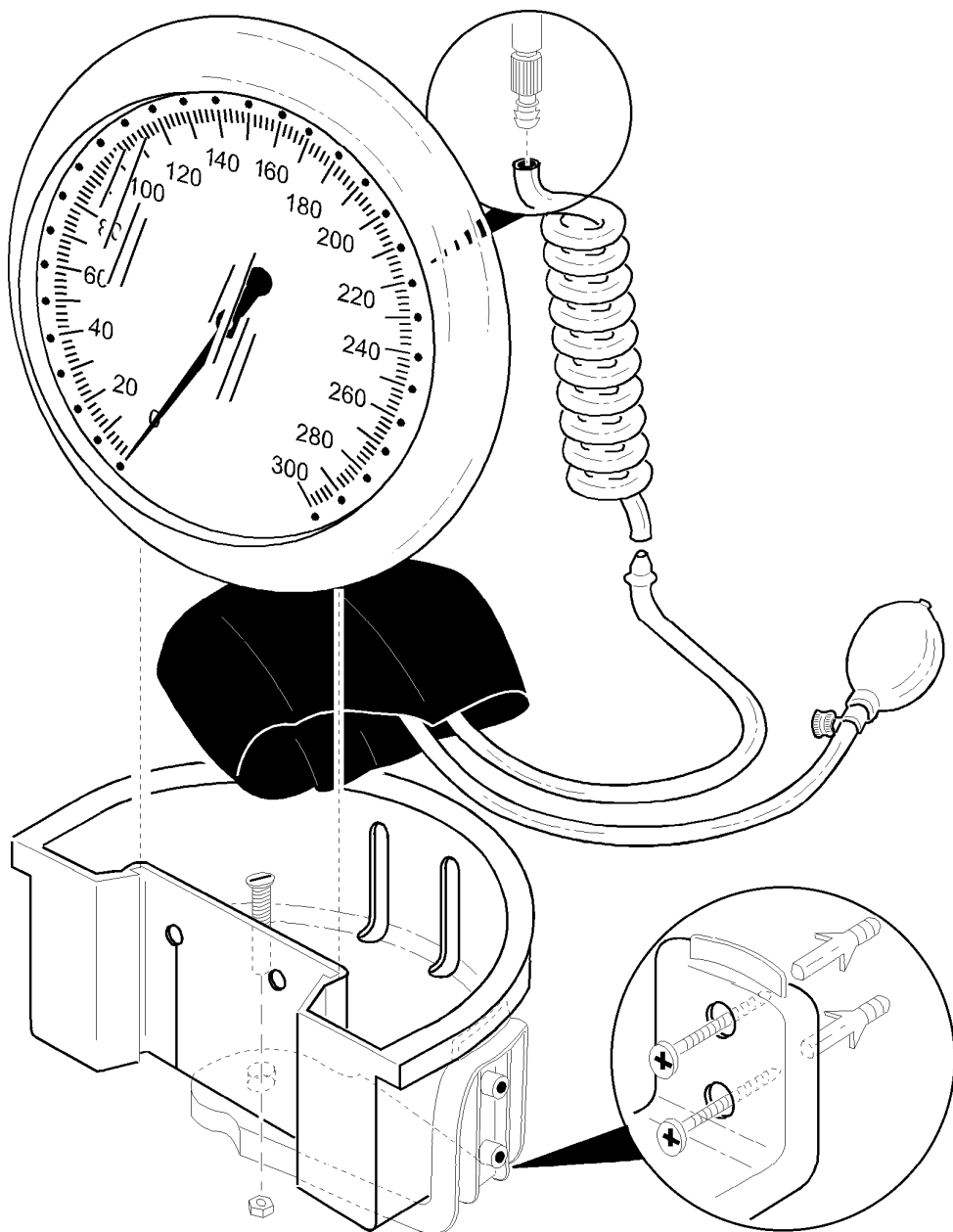
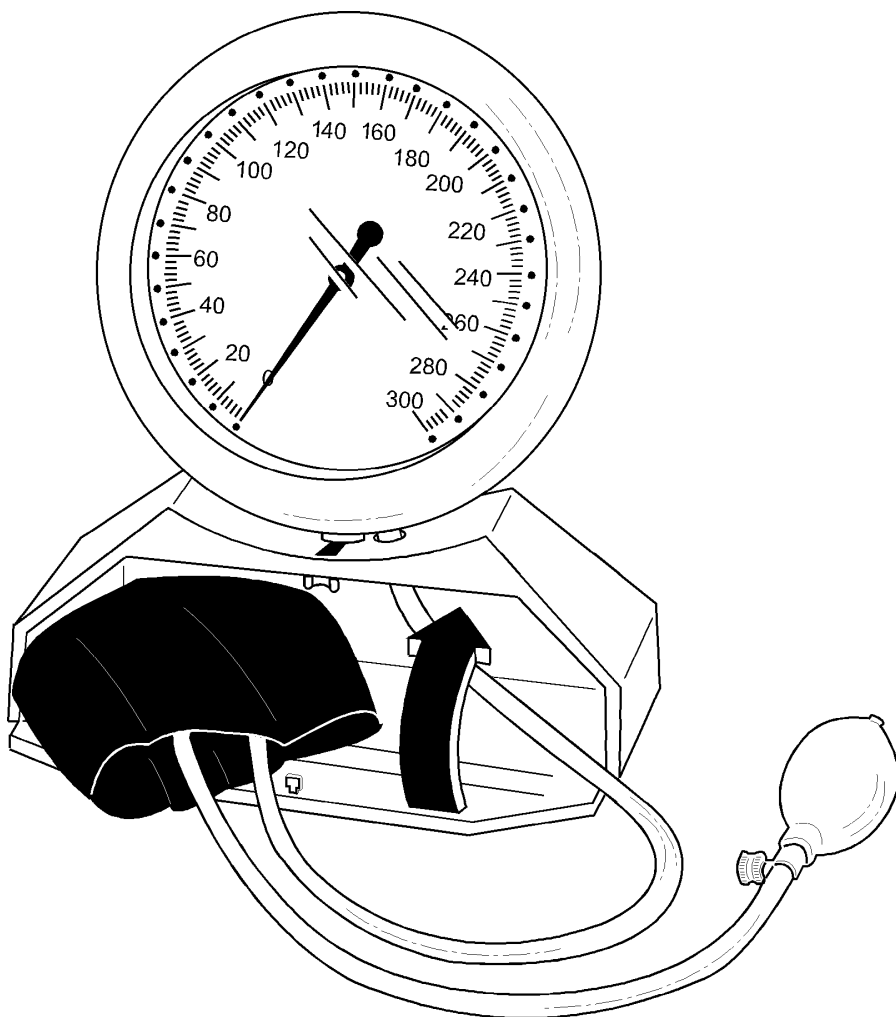
water and leave them to air dry. Nylon armbands must not be ironed.
The plenum chamber and tubes can be wiped with a damp cotton cloth.

GUARANTEE

The product has a 12 month warranty, valid from the date it is delivered by GIMA.
During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC. No compensation will be given for loss of use of the appliance.
Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.

The guarantee will be invalidated if the registration number has been removed, cancelled or altered. Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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