

**COLLETTORI GINECOLOGICI STERILI MONOUSO**  
**STERILE GYNAECOLOGICAL COLLECTORS FOR SINGLE USE**  
**COLLECTEURS GYNÉCOLOGIQUES STÉRILES À USAGE UNIQUE**  
**COLECTORES GINECOLÓGICOS ESTÉRILES DE UN SOLO USO**  
**ZARAGATOAS DE RECOLHA GINECOLÓGICAS ESTÉREIS DE UTILIZAÇÃO ÚNICA**  
**STERILE GYNÄKOLOGISCHE EINWEG-ABSTRICHBÜRSTEN**  
**ΑΠΟΣΤΕΙΡΩΜΕΝΟΥ ΓΥΝΑΙΚΟΛΟΓΙΚΟΥ ΣΥΜΛΕΚΤΗ ΜΙΑΣ ΧΡΗΣΗΣ**  
**STERYLNYCH KOLEKTORÓW GINEKOLOGICZNYCH JEDNORAZOWEGO UŻYTKU**  
**STERILNÍ GYNEKOLOGICKÉ ODBĚROVÉ POMŮCKY NA JEDNORÁZOVÉ POUŽITÍ**  
**STERILA GYNEKOLOGISKA UPSAMLINGSRÖR FÖR ENGÅNGSBRUK**  
**STERIILIN JA KERTAKÄYTTÖISTEN GYNEKOLOGIAN NÄYTETIKKUJEN**  
**STERILNE GINEKOLOŠKE KOLEKTORJE ZA ENKRATNO UPORABO**  
**JEDNORAZOVÉ STERILNÉ GYNEKOLOGICKÉ ZBERNÉ NÁDOBY**  
**COLECTOARELE GINECOLOGICE STERILE DE UNICĂ FOLOSINȚĂ**  
**STERIELE GYNAECOLOGISCHE UITSTRIJKBORSTELTJES VOOR EENMALIG GEBRUIK**  
**STERILNE GINEKOLOŠKE KOLEKTORE ZA JEDNOKRATNU UPORABU**  
**STERIL NŐGYÓGYÁSZATI KENETVÉTELI PÁLCÁHOZ**  
**STERILE GYNÆKOLOGISKE SAMLERE TIL ENGANGSBRUG**  
**СТЕРИЛНИ ГИНЕКОЛОГИЧНИ КОЛЕКТОРИ ЗА ЕДНОКРАТНА УПОТРЕБА**  
**VIENKARTINIO NAUDOJIMO STERILIŲ GINEKOLOGINIŲ MĖGINIŲ ĖMIMO PRIEMONIŲ**  
**VIENREIZĖJAI LIETOŠANAI PAREDŽĖTIEM STERILIEM GINEKOLOGIŠKIEM KOLEKTORIEM**  
**STERIILSE GŪNEKOLOGILISE PROOVIKOGUJA JUHEND**

مجمعات أمراض النساء المعقمة للاستخدام الواحد

**GIMA 29735 29736 29737 29745 29748 29749 29826 29827**



**Jiangsu Suyun Medical Materials Co., Ltd. No.18**  
Jin Qiao Road Dapu Industrial Park 222002 Lianyungang  
JIANGSU PROVINCE, PEOPLE'S REPUBLIC OF CHINA  
Made in China



**Medicare AG,**  
Hauptstr. 51 5024 Küttigen,  
Schweiz/CHRN-AR-20002506



**Shanghai International Holding Corp. GmbH (Eu-  
rope)**  
Eiffestraße, 80, 20537, Hamburg, Germany  
Tel. 0049-40-2513175



420211 IC - 4202221C - 47770111ID - 425002 - 420301 111B-3 - 420311 111B-4  
4204 11 - 4204 14



**Gima S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

    0123



**All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.**

**Product name:** Sterile Gynaecological Collectors for single use

**Indications:** This product is suitable for diseases that require gynecological examination.

**Intended purpose:** The product is used in gynecology and venereal disease.

**Contraindications:** None.

**Caution:**

A. When the product is used, it should strictly implement the requirements of aseptic operation specifications and related regulations, and be used by trained doctors and nurses. In the course of use, the operator or user's body should be treated in a timely manner according to medical regulations.

B. If you find any possible quality doubt during use, please stop using it immediately and notify our company quickly.

C. This product is used for single use and is destroyed after using; if the small package



is damaged and the product is prohibited to be used; this product is sterilized by ethylene oxide and the product is sterile; the product is valid for 5 years and should not be used after it expires.

D. Appropriate types and specifications should be selected according to the patient's body type.

E. If there are burrs, cracks, or nodes in the application area of the gynaecological collector, they should not be used.

F. This product shall be disposed of according to the requirements of waste disposal in the local medical regulations after using.

**Patient group:** it is suitable for people who require gynecological examination.

**Intended users:** It can be used by trained doctors, nursing staff.

**Clinical benefits:** The product plays an auxiliary role in the treatment of diseases.

**Any residual risk and any undesirable side-effects:** None

**Install information:** None

**Performance characteristics:** The product is provided in a sterile state and sterilized with ethylene oxide. Single-use

**Physical performance:**

a. The plastic part is complete, without obvious defects such as sharp edges and burrs, and there is no obvious bending phenomenon in the plastic part.

b. The surface is clean and free from obvious foreign objects and impurities.

**Structural composition:** The product is composed of handle and collector head etc.

**Method:**

a. Use the dilator to open the vagina at an appropriate angle.

b. Determine the location to be inspected or sampled.

c. Deepen the sampling head of the gynecological sampler to the sampling site and take a rotating or wiping flat method to sample.


**Storage:** The product should be stored in a well ventilated room





temperature.


### **GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.





**Indice dei simboli - Symbol index - Index des symboles**  
**- Índice de símbolos- índice de símbolo - Symbolindex -**  
**Ευρετήριο συμβόλων - Indeks symboli - Index symbolů**  
**- Symbol index - Symboli-indeksi - Indeks simbolov - Index**  
**symbolov - Index de simbol - Symbol index - Indeks simbo-**  
**la - Szimbólum index - Symbolindeks - Индекс на символа**  
**- Simbolių rodyklė - Simbolu rādītājs - Sūmbolite indeks**

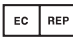

	<p> <b>IT</b> - Data di fabbricazione <b>GB</b> - Date of manufacture <b>FR</b> - Date de fabrication <b>ES</b> - Fecha de fabricación <b>PT</b> - Data de fabrico <b>DE</b> - Herstellungsdatum <b>GR</b> - Ημερομηνία παραγωγής <b>PL</b> - Data produkcji <b>CZ</b> - Datum výroby <b>SE</b> - Tillverkningsdatum <b>FI</b> - Valmistuspäivämäärä <b>SI</b> - Datum proizvodnje <b>SK</b> - Dátum výroby <b>RO</b> - Data fabricației <b>NL</b> - Productiedatum <b>HR</b> - Datum proizvodnje <b>HU</b> - Gyártás dátuma <b>DK</b> - Fabrikationsdato <b>BG</b> - Fabrikationsdato <b>LT</b> - Pagaminimo data <b>LV</b> - Izgatavošanas datums <b>EE</b> - Valmistamise kuupäev                 </p>
	<p> <b>IT</b> - Fabbricante <b>GB</b> - Manufacturer <b>FR</b> - Fabricant <b>ES</b> - Fabricante <b>PT</b> - Fabricante <b>DE</b> - Hersteller <b>GR</b> - Παραγωγός <b>PL</b> - Producent <b>CZ</b> - Výrobce <b>SE</b> - Tillverkare <b>FI</b> - Valmistaja <b>SI</b> - Proizvajalec <b>SK</b> - Výrobca <b>RO</b> - Producător <b>NL</b> - Fabrikant <b>HR</b> - Proizvođač <b>HU</b> - Gyártó <b>DK</b> - Fabrikant <b>BG</b> - Fabrikant <b>LT</b> - Gamintojas <b>LV</b> - Ražotājs <b>EE</b> - Tootja <b>NO</b> - ???                 </p>
	<p> <b>IT</b> - Conservare al riparo dalla luce solare <b>GB</b> - Keep away from sunlight <b>FR</b> - À conserver à l'abri de la lumière du soleil <b>ES</b> - Conservar al amparo de la luz solar <b>PT</b> - Guardar ao abrigo da luz solar <b>DE</b> - Vor Sonneneinstrahlung geschützt lagern <b>GR</b> - Κρατήστε το μακριά από ηλιακή ακτινοβολία <b>PL</b> - Przechowywać z dala od światła słonecznego <b>CZ</b> - Skladujte mimo sluneční světlo <b>SE</b> - Skyddas från solljus <b>FI</b> - Säilytä auringonvalolta suojassa <b>SI</b> - Hraniti zaščiteno pred sončno svetlobo <b>SK</b> - Skladujte mimo slnečného svetla <b>RO</b> - A se păstra ferit de razele soarelui <b>NL</b> - Afgeschermd van zonlicht opslaan <b>HR</b> - Čuvati zaštićeno od sunčeve svjetlosti <b>HU</b> - Napfénytől védve tárolandó <b>DK</b> - Må ikke udsættes for sollys <b>BG</b> - Må ikke udsættes for sollys <b>LT</b> - Saugoti nuo saulės spindulių <b>LV</b> - Uzglabāt prom no saules gaismas <b>EE</b> - Hoida eemal päikesevalgusest                 </p>

	<p><b>IT</b> - Non utilizzare se l'imballaggio è danneggiato <b>GB</b> - Don't use if package is damaged <b>FR</b> - Ne pas utiliser si le colis est endommagé <b>ES</b> - No usar si el paquete está dañado <b>PT</b> - Não use se o pacote estiver danificado <b>DE</b> - Nicht verwenden, wenn das Paket beschädigt ist <b>GR</b> - Μην το χρησιμοποιείτε αν η συσκευασία είναι κατεστραμμένη <b>PL</b> - Nie używać, jeśli opakowanie jest uszkodzone <b>CZ</b> - Nepoužívejte, pokud je obal poškozen <b>SE</b> - Använd inte en förpackning som är skadad <b>FI</b> - Ei saa käyttää, jos pakkaus on vaurioitunut <b>SI</b> - Ne uporabljajte, če je embalaža poškodovana <b>SK</b> - Nepoužívajte, ak je obal poškodený <b>RO</b> - A nu se utiliza dacă ambalajul este deteriorat <b>NL</b> - Niet gebruiken als de verpakking beschadigd is <b>HR</b> - Ne koristiti ako je pakiranje oštećeno <b>HU</b> - Ne használja, ha a csomagolás sérült <b>DK</b> - Må ikke bruges, hvis pakken er beskadiget <b>BG</b> - Må ikke bruges, hvis pakken er beskadiget <b>LT</b> - Nenaudokite, jei pakuotė pažeista <b>LV</b> - Nelietot, ja iepakojums ir bojāts <b>EE</b> - Ärge kasutage, kui pakend on kahjustatud</p>	 <p><b>IT</b> - Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso <b>GB</b> - Caution: read instructions (warnings) carefully <b>FR</b> - Attention: lisez attentivement les instructions (avertissements) <b>ES</b> - Precaución: lea las instrucciones (advertencias) cuidadosamente <b>PT</b> - Cuidado: leia as instruções (avisos) cuidadosamente <b>DE</b> - Achtung: Anweisungen (Warnungen) sorgfältig lesen <b>GR</b> - Προσοχή: διαβάστε προσεκτικά τις οδηγίες (ενοστάσεις) <b>PL</b> - Ostrzeżenie - Zobacz instrukcję obsługi <b>CZ</b> - Pozor: Pečlivě si přečtěte a dodržujte pokyny (varování) k použití <b>SE</b> - Varsamhet: läs anvisningarna (varningar) noga <b>FI</b> - Huomio: Lue käyttöohjeet (varoitukset) ja noudata niitä huolellisesti <b>SI</b> - Pozor: Preberite in skrbno sledite navodilom (opozorilom) za uporabo <b>SK</b> - Pozor: Pozorne si prečítajte a dodržívajte pokyny na použitie (výstrahy) <b>RO</b> - Atenție: Citiți și respectați cu atenție instrucțiunile (avertismentele) de utilizare <b>NL</b> - Opgelet: Lees en volg aandachtig de gebruiksaanwijzing (waarschuwingen) <b>HR</b> - Pozor: Pročitajte i pažljivo slijedite upute (upozorenja) za upotrebu <b>HU</b> - Figyelem: Figyelmesen olvassa el és kövesse a használati utasításokat (figyelmeztetéseket) <b>DK</b> - Forsigtig: Læs instruktationer (advarsler) omhyggeligt <b>BG</b> - Forsigtig: Læs instruktationer (advarsler) omhyggeligt <b>LT</b> - Dėmesio: perskaitykite ir atidžiai laikykites naudojimo instrukcijų (įspėjimų). <b>LV</b> - Uzmanību: Izlasiet un uzmanīgi ievērojiet lietošanas instrukcijas (brīdinājumus) <b>EE</b> - Tähelepanu! Lugege kasutusjuhised (hoiatused) läbi ja järgige neid hoolikalt</p>
	<p><b>IT</b> - Importato da <b>GB</b> - Imported by <b>FR</b> - Importé par <b>ES</b> - Importado por <b>PT</b> - Importado por <b>DE</b> - Eingeführt von <b>GR</b> - Εισαγωγή από <b>PL</b> - Importowane przez <b>CZ</b> - Dovezeno uživatelem <b>SE</b> - Importerad av <b>FI</b> - Tuojaa <b>SI</b> - Uvozil <b>SK</b> - Dovážal <b>RO</b> - Importat de <b>NL</b> - Geïmporteerd door <b>HR</b> - Uvezeno od strane <b>HU</b> - Importáltá <b>DK</b> - Importeret af <b>BG</b> - Importeret af <b>LT</b> - Importavo <b>LV</b> - Importēja <b>EE</b> - Importija</p>	
	<p><b>IT</b> - Conservare in luogo fresco ed asciutto <b>GB</b> - Keep in a cool, dry place <b>FR</b> - À conserver dans un endroit frais et sec <b>ES</b> - Conservar en un lugar fresco y seco <b>PT</b> - Armazenar em local fresco e seco <b>DE</b> - An einem kühlen und trockenen Ort lagern <b>GR</b> - Διατηρείται σε όρσοερό και στεγνό περιβάλλον <b>PL</b> - Przechowywać w suchym i mjejscu <b>CZ</b> - Skladujte na větraném a suchém místě <b>SE</b> - Förvara på svalt och torrt ställe <b>FI</b> - Säilytä kuivassa ja viileässä <b>SI</b> - Hraniti na suhem in hladnem mestu <b>SK</b> - Skladujte na chladnom a suchom mieste <b>RO</b> - A se păstra într-un loc răcoros și uscat <b>NL</b> - Koel en droog opslaan <b>HR</b> - Čuvati na hladnom i suhom mjestu <b>HU</b> - Száras, hűvös helyen tárolandó <b>DK</b> - Opbevares køligt og tørt <b>BG</b> - Opbevares køligt og tørt <b>LT</b> - Laikyti vėsioje ir sausoje vietoje <b>LV</b> - Uzglabāt vēsā, sausā vietā <b>EE</b> - Hoida jahedas ja kuivas kohas</p>	<p><b>IT</b> - Sterilizzato con ossido di etilene <b>GB</b> - Sterilized using ethylene oxide <b>FR</b> - Stérilisé à l'oxyde d'éthylène <b>ES</b> - Esterilizado con óxido de etileno <b>PT</b> - Esterilizado com óxido de etileno <b>DE</b> - Sterilisiert mit Ethylenoxid <b>GR</b> - Αποστειρωμένο με αιθυλενοξειδίο <b>PL</b> - Sterylizowane tlenkiem etylenu <b>CZ</b> - Sterilizováno etylenoxidem <b>SE</b> - Steriliserad med etylenoxid <b>FI</b> - Steriloitu etyleenioksidilla <b>SI</b> - Sterilizirano z etilen oksidom <b>SK</b> - Sterilizované etylenoxidom <b>RO</b> - Sterilizat cu oxid de etilenă <b>NL</b> - Gesteriliseerd met ethyleenoxide <b>HR</b> - Sterilizirano etilen oksidom <b>HU</b> - Etilén-oxidáal sterilizálva <b>DK</b> - Steriliseret med ethylenoxid <b>BG</b> - Steriliseret med ethylenoxid <b>LT</b> - Sterilizuotas etileno oksidu <b>LV</b> - Sterilizēts ar etilēnoksidu <b>EE</b> - Steriliseeritud etüleenioksiidiga</p> <p><b>STERILE</b></p>

<div data-bbox="67 414 140 454" data-label="Text">                 EC    REP             </div>	<p> <b>IT</b> - Rappresentante autorizzato nella Comunità europea <b>GB</b> - Authorized representative in the European community <b>FR</b> - Représentant autorisé dans la Communauté européenne <b>ES</b> - Representante autorizado en la Comunidad Europea <b>PT</b> - Representante autorizado na União Europeia <b>DE</b> - Autorisierter Vertreter in der EG <b>GR</b> - Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Ένωση <b>PL</b> - Upraważniony przedstawiciel we Wspólnocie Europejskiej <b>CZ</b> - Zplnomocněný zástupce v Evropském společenství <b>SE</b> - Auktoriserad representant i Europeiska gemenskapen <b>FI</b> - Valtuutettu edustaja Euroopan yhteisössä <b>SI</b> - Pooblaščen zastopnik za Evropsko skupnost <b>SK</b> - Splnomocnený zástupca v Európskom spoločenstve <b>RO</b> - Reprezentant autorizat pe teritoriul Comunității Europene <b>NL</b> - Bevoegde vertegenwoordiger in de Europese Gemeenschap <b>HR</b> - Ovlašteni predstavnik u Europskoj zajednici <b>HU</b> - Meghatalmazott képviselő az Európai Közösségben <b>DK</b> - Autoriseret repræsentant i det Europæiske Fællesskab <b>BG</b> - Autoriseret repræsentant i det Europæiske Fællesskab <b>LT</b> - Įgaliojatis atstovas Europos bendrijoje <b>LV</b> - Pilnvarotais pārstāvis Eiropas Kopienā <b>EE</b> - Volitatud esindaja Euroopa Ühenduses                 </p>
<div data-bbox="78 957 128 997" data-label="Image">  </div>	<p> <b>IT</b> - Leggere le istruzioni per l'uso <b>GB</b> - Consult instructions for use <b>FR</b> - Consulter les instructions d'utilisation <b>ES</b> - Consultar las instrucciones de uso <b>PT</b> - Consulte as instruções de uso <b>DE</b> - Gebrauchsanweisung beachten <b>GR</b> - Διαβάστε προσεκτικά τις οδηγίες χρήσης <b>PL</b> - Przeczytaj instrukcję użytkowania <b>CZ</b> - Přečtěte si návod k použití <b>SE</b> - Läs bruksanvisningen <b>FI</b> - Lue käyttöohjeet <b>SI</b> - Preberite navodila za uporabo <b>SK</b> - Prečítajte si návod na použitie <b>RO</b> - Citiți instrucțiunile de utilizare <b>NL</b> - Lees de gebruiksaanwijzing <b>HR</b> - Pročitajte upute za uporabu <b>HU</b> - Olvassa el a használati utasításokat <b>DK</b> - Se brugsvejledningen <b>BG</b> - Se brugsvejledningen <b>LT</b> - Perskaitykite naudojimo instrukcijas <b>LV</b> - Izlasiet lietošanas instrukcijas <b>EE</b> - Lugege kasutusjuhendit                 </p>

<div data-bbox="599 399 644 446" data-label="Image">  </div>	<p> <b>IT</b> - Dispositivo monouso, non riutilizzare <b>GB</b> - Disposable device, do not re-use <b>FR</b> - Dispositif pour usage unique, ne pas réutiliser <b>ES</b> - Dispositivo monouso, no reutilizable <b>PT</b> - Dispositivo descartável, não reutilizar <b>DE</b> - Für einmaligen Gebrauch, nicht wiederverwenden <b>GR</b> - Προϊόν μιας χρήσεως. Μην το χρησιμοποιείτε εκ νέου <b>PL</b> - Jedno użyczenie, nie używaj ponownie <b>CZ</b> - Jednorázový prostředek, nepoužívejte opakovaně <b>SE</b> - Engångsanordning, får ej återanvändas <b>FI</b> - Kertakäyttöinen laite, ei saa käyttää uudelleen <b>SI</b> - Za enkratno uporabo, ne uporabiti ponovno <b>SK</b> - Zariadenie na jedno použitie, nepoužívajte opakovane <b>RO</b> - Dispozitiv de unică folosință, a nu se refolosi <b>NL</b> - Voor eenmalig gebruik, niet hergebruiken <b>HR</b> - Uredaj za jednokratnu upotrebu, nemojte ponovo koristiti <b>HU</b> - Eldobható eszköz, ne használja újra <b>DK</b> - Engangsenhed, må ikke genbruges <b>BG</b> - Engangsenhed, må ikke genbruges <b>LT</b> - Vienkartinis prietaisas, nenaudokite pakartotinai <b>LV</b> - Vienreiz lietojama ierīce, nelietojiet to atkārtoti <b>EE</b> - Ühekordne, ärge kasutage seda mitu korda                 </p>
<div data-bbox="588 1101 660 1157" data-label="Image">  </div>	<p> <b>IT</b> - Dispositivo medico conforme al regolamento (UE) 2017/745 <b>GB</b> - Medical device compliant with Regulation (EU) 2017/745 <b>FR</b> - Dispositif médical conforme au Règlement (UE) 2017/745 <b>DE</b> - Medizinprodukt gemäß Verordnung (EU) 2017/745 <b>ES</b> - Dispositivo médico que cumple con el Reglamento (UE) 2017/745 <b>PT</b> - Dispositivo médico em conformidade com o Regulamento (UE) 2017/745 <b>GR</b> - Ιατροτεχνολογικό προϊόν συμμορφούμενο με τον Κανονισμό (ΕΕ) 2017/745 <b>BG</b> - Медицинско изделие, отговарящо на Регламент (ЕС) 2017/745 <b>CZ</b> - Zdravotnický prostředek v souladu s nařízením (EU) 2017/745 <b>DA</b> - Medicinsk udstyr i overensstemmelse med forordning (EU) 2017/745 <b>EE</b> - Määrusele (EL) 2017/745 vastav meditsiiniseade <b>FI</b> - Asetuksen (EU) 2017/745 mukainen lääkinällinen laite <b>HR</b> - Medicinski uređaj usklađen s Uredbom (EU) 2017/745 <b>HU</b> - Az (EU) 2017/745 rendeletnek megfelelő orvostechnikai eszköz <b>LT</b> - Medicinos prietaisas, atitinkantis Reglamentą (ES) 2017/745 <b>LV</b> - Medicīnas ierīce, kas atbilst Regulai (ES) 2017/745 <b>NO</b> - Medisinsk utstyr i samsvar med forordning (EU) 2017/745 <b>NL</b> - Medisch hulpmiddel dat voldoet aan Verordening (EU) 2017/745 <b>PL</b> - Wyrób medyczny zgodny z Rozporządzeniem (UE) 2017/745 <b>RO</b> - Dispozitiv medical conform Regulamentului (UE) 2017/745 <b>SK</b> - Zdravotnícka pomôcka v súlade s nariadením (EÚ) 2017/745 <b>SL</b> - Medicinski pripomoček skladen za Uredbo (EU) 2017/745 <b>SV</b> - Medicinsk utrustning i enlighet med förordning (EU) 2017/745                 </p>

	<p><b>IT</b> - Data di scadenza <b>GB</b> - Expiration date <b>FR</b> - Date d'échéance <b>ES</b> - Fecha de caducidad <b>PT</b> - Data de validade <b>DE</b> - Ablaufdatum <b>GR</b> - Ημερομηνία λήξεως <b>PL</b> - Data ważności <b>CZ</b> - Datum ukončení platnosti <b>SE</b> - Utgångsdatum <b>FI</b> - Vimeinen voimassaolopäivä <b>SI</b> - Rok uporabnosti <b>SK</b> - Dátum expirácie <b>RO</b> - Valabil până la data de <b>NL</b> - Vervaldatum <b>HR</b> - Datum isteka <b>HU</b> - Lejárati dátum <b>DK</b> - Udløbsdato <b>BG</b> - Udløbsdato <b>LT</b> - Galiojimo laikas <b>LV</b> - Derīguma termiņš <b>EE</b> - Aegumiskuupäev</p>
	<p><b>IT</b> - Codice prodotto <b>GB</b> - Product code <b>FR</b> - Code produit <b>ES</b> - Código producto <b>PT</b> - Código produto <b>DE</b> - Erzeugniscode <b>GR</b> - Κωδικός προϊόντος <b>PL</b> - Numer katalogowy <b>CZ</b> - Kód výrobku <b>SE</b> - Produktkod <b>FI</b> - Tuotekoodi <b>SI</b> - Koda izdelka <b>SK</b> - Kód výrobku <b>RO</b> - Cod produs <b>NL</b> - Productcode <b>HR</b> - Šifra proizvoda <b>HU</b> - Termékkód <b>DK</b> - Produktkode <b>BG</b> - Produktkode <b>LT</b> - Prekės kodas <b>LV</b> - Produkta kods <b>EE</b> - Toote kood</p>
	<p><b>IT</b> - Numero di lotto <b>GB</b> - Lot number <b>FR</b> - Numéro de lot <b>ES</b> - Número de lote <b>PT</b> - Número de lote <b>DE</b> - Chargennummer <b>GR</b> - Αριθμός παρτίδας <b>PL</b> - Kod partii <b>CZ</b> - Číslo šarže <b>SE</b> - Satsnummer <b>FI</b> - Eränumero <b>SI</b> - Številka partije <b>SK</b> - Číslo šarže <b>RO</b> - Număr de lot <b>NL</b> - Partijnummer <b>HR</b> - Broj serije <b>HU</b> - Tételszám <b>DK</b> - Batchnummer <b>BG</b> - Batchnummer <b>LT</b> - Partijos numeris <b>LV</b> - Partijas numurs <b>EE</b> - Partii number</p>
	<p><b>IT</b> - Non ri-sterilizzare <b>GB</b> - Do not resterilize <b>FR</b> - Ne pas resteriliser <b>ES</b> - No reesterilizar <b>PT</b> - Não reesterilize <b>DE</b> - Nicht resterilisieren <b>GR</b> - Μην αποστειρώσετε <b>PL</b> - Nie sterylizuj ponownie <b>CZ</b> - Sterilizaci neopakujte <b>SE</b> - Återsterilisera inte <b>FI</b> - Uudelleensterilointi kielletty <b>SI</b> - Ne sterilizirajte ponovno <b>SK</b> - Opakovane nesterilizujte <b>RO</b> - A nu se resteriliza <b>NL</b> - Niet hersteriliseren <b>HR</b> - Nemojte ponovno sterilizirati <b>HU</b> - Ne sterilizálja újra <b>DK</b> - Må ikke generiseres <b>BG</b> - Да не се стерилизира повторно <b>LT</b> - Nesterilizuoti pakartotinai <b>LV</b> - Neveikt atkārtotu sterilizāciju <b>EE</b> - Ärge steriliseerige uuesti</p>

	<p><b>IT</b> - Singolo sistema di barriera sterile con imballo protettivo esterno <b>GB</b> - Single sterile barrier system in protective outer packaging <b>FR</b> - Système de barrière stérile unique dans un emballage extérieur protecteur <b>ES</b> - Sistema de barrera estéril simple en embalaje exterior protector <b>PT</b> - Sistema de barreira estéril única em embalagem externa protetora <b>DE</b> - Einzelnes Sterilbarrieresystem in schützender Umverpackung <b>GR</b> - Μovό αποστειρωμένο σύστημα φραγής σε προστατευτική εξωτερική συσκευασία <b>PL</b> - Pojedynczy sterylony system barierowy w ochronnym opakowaniu zewnętrznym <b>CZ</b> - Jediný sterilní bariérový systém v ochranném vnějším obalu <b>SE</b> - Enkelt steril barriärsystem i skyddande ytterförpackning <b>FI</b> - Yksi steriili sulkujärjestelmä suojaavassa ulkopakkauksessa <b>SI</b> - Enojni sterilni pregradni sistem v zaščitni zunanji embalaži <b>SK</b> - Jednoduchý sterilný bariérový systém v ochrannom vonkajšom obale <b>RO</b> - Sistem de barieră steril unic în ambalaj exterior de protecție <b>NL</b> - Enkelvoudig steriel barriërsysteem in beschermende buitenverpakking <b>HR</b> - Jednostruki sustav sterilne barijere u zaštitnom vanjskom pakiranju <b>HU</b> - Egyetlen steril gátrendszer védő külső csomagolásban <b>DK</b> - Enkelt steril barrieresystem i beskyttende ydre emballage <b>BG</b> - Единична стерилна бариерна система в защитна външна опаковка <b>LT</b> - Viena sterilii barjerinė sistema apsauginėje išorinėje pakuotėje <b>LV</b> - Viena sterila barjeras sistēma aizsargājošā ārējā iepakojumā <b>EE</b> - Ühekordne steriilne tõkkesüsteem kaitsvas välispakendis</p>
	<p><b>IT</b> - Identificatore univoco del dispositivo <b>GB</b> - Unique device identifier <b>FR</b> - Identifiant unique de l'appareil <b>ES</b> - Identificador de dispositivo único <b>PT</b> - Identificador exclusivo do dispositivo <b>DE</b> - Unique Device Identifier (Eindeutige Kennung des Geräts) <b>GR</b> - Μοναδικό αναγνωριστικό συσκευής <b>PL</b> - Unikalny identyfikator urządzenia <b>CZ</b> - Jedinečný identifikátor zařízení <b>SE</b> - Unik identifierare för enheten <b>FI</b> - Laitteen yksilöllinen tunniste <b>SI</b> - Enolični identifikator naprave <b>SK</b> - Jedinečný identifikátor zariadenia <b>RO</b> - Identicatorul unic al dispozitivului <b>NL</b> - Unieke identificatie van het apparaat <b>HR</b> - Jedinstveni identifikator uređaja <b>HU</b> - Az eszköz egyedi azonosítója <b>DK</b> - Unik identifikator for enheden <b>BG</b> - Уникален идентификатор на устройството <b>LT</b> - Unikalus įrenginio identifikatorius <b>LV</b> - Unikāls ierīces identifikators <b>EE</b> - Seadme kordumatu identifikaator</p>

<div data-bbox="69 322 140 352" style="border: 1px solid black; padding: 2px; display: inline-block;">                 CH REP             </div>	<p> <b>IT</b> - Rappresentante autorizzato in Svizzera <b>GB</b>                      - Authorized Representative in Swiss <b>FR</b> -                      Représentant autorisé au Suisse <b>ES</b> -                      Representante autorizado en Suizo <b>PT</b> -                      Representante autorizado no Suíço <b>DE</b>                      - Autorisierter Vertreter im Schweizer <b>GR</b> -                      Εξουσιοδοτημένος Αντιπρόσωπος στο Ελβετός  <b>PL</b> - Autoryzowany Przedstawiciel w                      Szwajcarski <b>CZ</b> - Autorizovaný zástupce ve                      švýcarský <b>SE</b> - Auktoriserad representant i                      Schweiziska <b>FI</b> - Valtuutettu edustaja                      Sveitsiläinen <b>SI</b> - Pooblaščeni zastopnik v                      Švicaru <b>SK</b> - Autorizovaný zástupca v                      švajčiarsky <b>RO</b> - Reprezentant autorizat în                      Elvețiane <b>NL</b> - Geautoriseerde <b>HR</b> - Ovlašteni                      predstavnik u Zvitsers <b>HU</b> - Meghatalmazott                      képviselő az Svájci <b>DK</b> - Autoriseret                      repræsentant i Schweizisk <b>NO</b> - Autorisert                      representant i Sveitsiska <b>BG</b> - Оторизиран                      представител в швейцарски <b>LT</b> - Įgaliojas                      atstovas Šveicarijoje <b>LV</b> - Pilnvarotais pārstāvis                      -Šveices <b>EE</b> Pilnvarotais pārstāvis Šveicē                 </p>
<div data-bbox="82 746 127 772" style="border: 1px solid black; padding: 2px; display: inline-block;">                 MD             </div>	<p> <b>IT</b> - Dispositivo medico <b>GB</b> - Medical Device  <b>FR</b> - Dispositif médical <b>ES</b> - Producto                      sanitario <b>PT</b> - Dispositivo médico <b>DE</b> -                      Medizinprodukt <b>GR</b> - Ιατροτεχνολογικό                      προϊόν <b>PL</b> - Wyrób medyczny <b>CZ</b> -                      Zdravotnický prostředek <b>SE</b> - Medicinteknisk                      produkt <b>FI</b> - Lääkinnällinen laite <b>SI</b> -                      Medicinski pripomoček <b>SK</b> - Zdravotnícka                      pomôcka <b>RO</b> - Dispozitiv medical <b>NL</b> -                      Medisch hulpmiddel <b>HR</b> - Medicinski                      uređaj <b>HU</b> - Orvosteknikai eszköz <b>DK</b> -                      Medicinsk udstyr <b>BG</b> - Medicinsk udstyr <b>LT</b> -                      Medicininis prietaisas <b>LV</b> - Mediciniskā ierīce  <b>EE</b> - Meditsiiniline seade                 </p>



