GIMA

PROFESSIONAL MEDICAL PRODUCTS

APPLICATORE CON PUNTA IN COTONE
APPLICATOR WITH COTTON TIP
APPLICATEUR AVEC EMBOUT EN COTON
APLICADOR CON PUNTA DE ALGODÓN
APLICADOR COM PONTA DE ALGODÃO
APPLIKATOR MIT WATTESPITZE
ΑΠΛΙΚΑΤΕΡ ΜΕ ΒΑΜΒΑΚΕΡΗ ΑΚΡΗ
APLIKATOR Z BAWĘLNIANĄ KOŃCÓWKĄ
APLIKÁTOR S BAVLNENÝM HROTOM
APPLIKATOR MED BOMULLSTOPP
APPLIKAATTORI PUUVILLAKÄRJELLÄ
APLIKATOR Z VATIRANO KONICO
APLIKÁTOR S VATOVOU ŠPIČKOU
APLICATOR CU VÂRF DIN VATĂ
ΑΠΛΙΚΑΤΟΡ Σ ΠΑΜΥΧΕΗ ΗΑΚΡΑΪΗΝΙΚ
APPLICATOR MET KATOENEN TIP
APLIKATOR S PAMUČNIM VRHOM
APPLIKÁTOR PAMUT VATTAFEJJEL
APPLIKATOR MED BOMULDSSPIDS
APLIKATORIUS MEDVILNINIU GALIUKU
APLIKATORS AR KOKVILNAS UZGALI
VATIOTSAGA APLIKAATOR

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Jiangsu Suyun Medical Materials Co., Ltd. No.18
No.18 Jin Qiao Road Dapu Industrial Park 222002 Lianyungang,
Jiangsu province PEOPLE'S REPUBLIC OF CHINA
Made in China

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Contraindications: None.

Caution:

- A. Discard after for single use.
- B. Use by 5 years, prohibit use if overdue.
- C. The product should be stored in area of dry, aeration and clean.

Patient group: It is suitable for people who require cleaning and disinfection of skin or trauma.

Intended users: It can be used by trained personnel.

Clinical benefits: The product plays an auxiliary role in the treatment of diseases.

Any residual risk and any undesirable side-effects: none

Install information: none

Physical performance:

- a. The entire surface of the product should be clean and free from any foreign objects, impurities, stains, etc.
 - b. The cotton swab rod should be neatly cut without chips, notches, cracks, and burrs; Wooden poles should be free from insect infestation.
 - c. The degreased cotton head should be round and free from impurities, shedding, and other phenomena.
 - d. The pole and the absorbent cotton should be firmly connected, and the pole should be fixed.
- Apply 2 N axial tension to the absorbent cotton for 15 seconds, and the absorbent cotton should not fall off.

Structural composition: The product are composed of pole and absorbent cotton head.

Method:

- a. Open individual package by indication on package, take out the device.
- b. To use for disinfecting or scrubbing away dressing wound smearing liquid medicine etc for diseases skins.

Storage: The product should be stored in a well-ventilated room temperature.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

ENGLISH

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Product Name: Applicator with Cotton Tip

Type: I-1, I-2, I-3, I-4, I-5, I-6, I-7, II-1, II-2, II-3, II-4, II-5.

This product is suitable for disinfection of external injuries, knife injuries, burns, or surface disinfection of general items.

Intend purpose: The main clinical usage is to disinfect or scrub away dressing wound smear liquid medicine etc for diseases skins.

Indice dei simboli - Symbol index - Index des symboles - Índice de símbolos - índice de símbolo - Symbolindex - Ευρετήριο συμβόλων - Indeks symboli - Index symbolů - Symbol index - Symboli-indeksi - Indeks simbolov - Index symbolov - Index de simbol - Symbol index - Indeks simbola - Szimbólum index - Symbolindeks - Indeks over symboler - Индекс на симбола - Simbolių rodyklė - Simbolu rādītājs - Sümboleite indeks

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