



GIMA

PROFESSIONAL MEDICAL PRODUCTS

STETOSCOPI GIMA - GIMA STETHOSCOPE
STÉTHOSCOPE BIAURICULAIRES GIMA
ESTETOSCOPIOS GIMA - ESTETOSCÓPIO GIMA
GIMA STETHOSKOPE - GIMA SZTETOSZKÓP
STETOSKOPY GIMA - GIMA STETOSKOP
ΣΤΗΘΟΣΚΟΠΙΑ GIMA - STETOSCOAPE GIMA
GIMA STETOSKOPI - GIMA STETOSKOOPIT
ΣΤΕΤΟΣΚΟΠΙ GIMA

السماعة الطبية

È necessario segnalare qualsiasi incidente grave verificatosi in relazione al dispositivo medico da noi fornito al fabbricante e all'autorità competente dello Stato membro in cui si ha sede.

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

Il est nécessaire de signaler tout accident grave survenu et lié au dispositif médical que nous avons livré au fabricant et à l'autorité compétente de l'état membre où on a le siège social.

Es necesario informar al fabricante y a la autoridad competente del Estado miembro en el que se encuentra la sede sobre cualquier incidente grave que haya ocurrido en relación con el producto sanitario que le hemos suministrado.

É necessário notificar ao fabricante e às autoridades competentes do Estado-membro onde ele está sediado qualquer acidente grave verificado em relação ao dispositivo médico fornecido por nós.

Jeder schwere Unfall im Zusammenhang mit dem von uns gelieferten medizinischen Gerät muss unbedingt dem Hersteller und der zuständigen Behörde des Mitgliedsstaats, in dem das Gerät verwendet wird, gemeldet werden.

A gyártónak, illetve a székhely szerinti tagállam illetékes hatóságának jelezni kell bármilyen olyan súlyos balesetet, amely az általunk szállított orvostechnikai eszközzel kapcsolatban történt.

Należy poinformować producenta i kompetentne władze danego Kraju członkowskiego o każdym poważnym wypadku związanym z wyrobem medycznym naszej produkcji.

Det är nödvändigt att meddela tillverkaren och de behöriga myndigheterna i den berörda medlemsstaten, om alla allvarliga olyckor som inträffat i samband med den medicintekniska utrustning som levererats av oss.

Σε περίπτωση που διαπιστώσετε οποιοδήποτε σοβαρό περιστατικό σε σχέση με την ιατρική συσκευή που σας παρέχουμε θα πρέπει να το αναφέρετε στον κατασκευαστή και στην αρμόδια αρχή του κράτους μέλους στο οποίο βρίσκεστε.

Orice accident grav produs, privitor la dispozitivul medical fabricat de firma noastra, trebuie semnalat producătorului și autorității competente în statul membru pe teritoriul căruia își are sediul utilizatorul

Potrebno je prijaviti svaku ozbiljnu nezgodu koja se dogodila u vezi s isporučenim medicinskim proizvodaču i nadležnom tijelu države članice u kojoj se nalazi.

Kaikista vakavista tapaturmista, jotka liittyvät toimittamamme lääkinällisen laitteen käyttöön, on ilmoitettava valmistajalle sekä oman asuinpaikan jäsenmaan toimivaltaiselle viranomaiselle

Всички сериозни инциденти, които са настъпили във връзка с доставеното от нас медицинско изделие, трябва да се сигнализират на производителя и на компетентния орган на държавата членка, в която производителят е установен

يجب الإبلاغ فوراً عن أي حادث خطير وقع فيما يتعلق بالجهاز الطبي الذي زودنا به إلى الجهة الصانعة والسلطة المختصة في الدولة العضو التي يقع فيها

REF

32506 - 32508 - 32509 - 32511 - 32513 - 32515 - 32526 32527 - 32528 32542 -
 32569 - 32570 - 32571 - 32572 32574 - 32575 - 32576 - 32577
 49500 - 49501 - 49502 - 49503 - 49510 - 49511 - 49512 - 49513



Gima S.p.A.
 Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
 Made in Taiwan






These medical devices are intended for auscultation.

Bow adjustment

Position the eartubes so that they are inclined at about 15° and the eartips are in line with the nose bridge, facing forward with respect to you (see illustration).

In this way the eartubes will be aligned with the ear canal and you will obtain a clear, sharp sound.

Holding the eartubes in front of you with one tube in each hand, open them, separating the tubes and inserting the eartips in your ears.

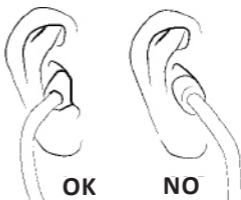

Adjustment of the bow tension

If you feel uncomfortable or performance is poor, adjust the tension of the eartubes. Press the binaural eartubes against each other gently to increase the tension, separate them to reduce it.



- **Attention: an excessive tension could weaken the bow.**
- **For best performance, it is important to use eartips of a suitable size for your ears. This is particularly important if you're using soft conformable eartips.**

Replacement of the diaphragm

Although the diaphragm is solid and planned to last long, it could be necessary to replace it.

In this case proceed as follows:

- Remove the backing ring, paying attention not to damage it (for screw models, counter clockwise rotate the ring), then pull the diaphragm out.
- Fit the new diaphragm and reposition the backing ring making sure that it is well fixed (for screw models clockwise rotate the ring).



Selection of frequency / cushion

for double-head models only

Rotating the chestpiece by 180°, it is possible to change the listening mode by opening (or indexing) the side you wish to use, either the diaphragm or open bell: select the diaphragm to listen to high frequencies, or the open bell to listen to low frequencies. "If the diaphragm is open, the bell will be closed, blocking the sound coming from the bell and vice versa. To determine which of the two sides is active, position the eartips in your ears and exert slight pressure on the diaphragm or on the hole of the bell.

Cleaning and maintenance

Regular and periodic cleaning will ensure best performance over time:



- Keep away from extreme heat and cold, solvents and oils.
- Do not submerge the product in liquids and do not subject it to sterilisation processes.
- Disinfect the product after each use, using a 70% isopropyl alcohol solution.
- Avoid damage to the product caused by sharp objects.
- Regularly unscrew the eartips to keep them clean. Clean the eartips (and various parts of the product, if necessary) with alcohol or soapy water and dry them completely before re-assembling them. Make sure the eartips have been securely refitted to the ends of the eartubes.
- Do not use the product disassembled (e.g. without eartips) and/or in the event of visible damage

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.



	<p>IT - Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso GB - Caution: read instructions (warnings) carefully FR - Precaución: lea las instrucciones (advertencias) cuidadosamente ES - Precaución: lea las instrucciones (advertencias) cuidadosamente PT - Cuidado: leia as instruções (avisos) cuidadosamente DE - Achtung: Anweisungen (Warnungen) sorgfältig lesen HU - Figyelem: Figyelmesen olvassa el és kövesse a használati utasításokat (figyelmeztetéseket) PL - Ostrzeżenie – Zobacz instrukcję obsługi SE - Varsamhet: läs anvisningarna (varningar) noga GR - Προσοχή: διαβάστε προσεκτικά τις οδηγίες (ενοτάσεις) RO - Atenție: Citiți și respectați cu atenție instrucțiunile (avertismentele) de utilizare HR - Pozor: Pročitajte i pažljivo slijedite upute (upozorenja) za upotrebu FI - Huomio: Lue käyttöohjeet (varoitukset) ja noudata niitä huolellisesti BG - Forsigtig: Læs instruktioner (advarsler) omhyggeligt</p> <p>SA - الحذر: قراءة التعليمات (التحذيرات) بعناية</p>
	<p>IT - Conservare in luogo fresco ed asciutto GB - Keep in a cool, dry place FR - Á conserver dans un endroit frais et sec ES - Conservar en un lugar fresco y seco PT - Armazenar em local fresco e seco DE - An einem kühlen und trockenen Ort lagern HU - Száras helyen tartandó PL - Przechowywać w suchym miejscu SE - Förvara på svalt och torrt ställe GR - Διατηρείται σε δροσερό και στεγνό περιβάλλον RO - A se păstra într-un loc răcoros și uscat HR - Čuvati na hladnom i suhom mjestu FI - Säilytä kuivassa ja viileässä BG - Opbevaras køligt og tørt</p> <p>SA - يحفظ في مكان بارد وجاف</p>
	<p>IT - Conservare al riparo dalla luce solare GB - Keep away from sunlight FR - Á conserver à l'abri de la lumière du soleil ES - Conservar al amparo de la luz solar PT - Guardar ao abrigo da luz solar DE - Vor Sonneneinstrahlung geschützt lagern HU - Napfénytől védve tárolandó PL - Przechowywać z dala od światła słonecznego SE - Skyddas från solljus GR - Κρατήστε το μακριά από ηλιακή ακτινοβολία RO - A se păstra ferit de razele soarelui HR - Čuvati zaštićeno od sunčeve svjetlosti FI - Säilytä auringonvalolta suojassa BG - Må ikke udsættes for sollys</p> <p>SA - يحفظ بعيداً عن ضوء الشمس</p>
	<p>IT - Fabbricante GB - Manufacturer FR - Fabricant ES - Fabricante PT - Fabricante DE - Hersteller HU - Gyártó PL - Producent SE - Tillverkare GR - Παραγωγός RO - Producător HR - Proizvođač FI - Valmistaja BG - Fabrikant</p> <p>SA - الشركة المصنعة</p>
	<p>IT - Leggere le istruzioni per l'uso GB - Consult instructions for use FR - Consulter les instructions d'utilisation ES - Consultar las instrucciones de uso PT - Consulte as instruções de uso DE - Gebrauchsanweisung beachten HU - Olvassa el a használati utasítást! PL - Przeczytaj instrukcję użytkowania SE - Läs bruksanvisningen GR - Διαβάστε προσεκτικά τις οδηγίες χρήσης RO - Citiți instrucțiunile de utilizare HR - Pročitajte upute za uporabu FI - Lue käyttöohjeet BG - Se brugsvejledningen</p> <p>SA - اقرأ بدقة وحرص تعليمات الاستخدام</p>
	<p>IT - Dispositivo medico conforme al regolamento (UE) 2017/745 GB - Medical Device compliant with Regulation (EU) 2017/745 FR - Dispositif médical conforme au règlement (UE) 2017/745 ES - Producto sanitario conforme con el reglamento (UE) 2017/745 PT - Dispositivo médico em conformidade com a regulamento (UE) 2017/745 DE - Medizinprodukt im Sinne der Verordnung (EU) 2017/745 HU - A 2017/745/EU rendeletnek megfelelő orvostechnikai eszköz PL - Wyrób medyczny zgodny z Rozporządzenie (UE) 2017/745 SE - Den medicintekniska produkten överensstämmer med förordning (EU) 2017/745 GR - Ιατρική συσκευή σύμφωνα με την ΚΑΝΟΝΙΣΜΟΣ (ΕΕ) 2017/745 RO - Dispozitiv medical conform regulamentului (UE) 2017/745 HR - Medicinski proizvod sukladan propisu (EU) 2017/745 FI - Lääkinnällinen laite, joka vastaa asetusta (EU) 2017/745 BG - Medicinsk udstyr i overensstemmelse med forordning (EU) 2017/745</p> <p>SA - جهاز طبي يتوافق مع التوجيه (UE) 2017/745</p>



REF	<p>IT - Codice prodotto GB - Product code FR - Code produit ES - Código producto PT - Código produto DE - Erzeugniscod HU - Termékkód PL - Numer katalogowy SE - Produktkod GR - Κωδικός προϊόντος RO - Cod produs HR - Šifra proizvoda FI - Tuotekoodi BG - Produktkode</p> <p style="text-align: right;">SA - كود المنتج</p>
LOT	<p>IT - Numero di lotto GB - Lot number FR - Numéro de lot ES - Número de lote PT - Número de lote DE - Chargennummer HU - Tételszám PL - Kod partii SE - Satsnummer GR - Αριθμός παρτίδας RO - Număr de lot HR - Broj serije FI - Eränumero BG - Batchnummer</p> <p style="text-align: right;">SA - رقم الدفعة</p>
MD	<p>IT - Dispositivo medico GB - Medical Device FR - Dispositif médical ES - Producto sanitario PT - Dispositivo médico DE - Medizinprodukt HU - Orvostechikai eszköz PL - Wyrób medyczny SE - Medicinteknisk produkt GR - Ιατροτεχνολογικό προϊόν RO - Dispozitiv medical HR - Medicinski uređaj FI - Lääkinnällinen laite BG - Medicinsk udstyr</p> <p style="text-align: right;">SA - جهاز طبي</p>