

## **INSTRUCTIONS FOR USE**

### **FLY/SAFE SAFETY SCALP VEIN SETS**

#### **INTENDED USE**

For single use, sterile, non-toxic, non-pyrogenic, safety scalp vein sets for blood sampling or vein injections.

#### **INSTRUCTIONS FOR USE:**

Before using check the expiry date. Do not use if packaging is damaged. In case of re-use, infections can occur: consult a doctor.

Prepare the area carefully and aseptically. Remove the scalp vein set from the sterile packaging. Remove the needle cap. Avoid any contact with the needle. Go on with the use. After the process, put the fingers on the graining and make the safety mechanism slide ahead, until a "click" is heard. It means that the needle has been blocked into the protection irreversibly. Dispose of it properly after use.

#### **LATEX FREE:**

The device is latex free in order to prevent any risk of allergic reaction.

#### **PHTHALATE:**

The device is DEHP free.

#### **STERILIZATION METHOD:**

Sterilized by Ethylene Oxide (EO), residual amount of gas in accordance with the regulations.

#### **DISPOSAL:**

Disposal must be made in accordance with the applicable government regulations. Incineration or landfill disposal must be carried out under controlled conditions. The product that did not come into contact with infectious agents or other hazardous materials, can not be considered dangerous waste.

#### **CONSERVATION:**

Store in a cool dry place and far away from the any source of heat at a temperature.

#### **MANUFACTURER:**

RAYS SPA  
VIA FRANCESCO CRISPI  
60027 OSIMO (AN) – IT

**EC Certificate:** EC Certificate n. IT268973 issued by Notified Body n. 1370 – Bureau Veritas Italia Spa – released to RAYS S.p.A. and valid until 13/12/2022 in compliance with the Directive 93/42/EEC requirements, attachment V.

*These Instructions for Use note must be preserved by the user for the duration of the MD.*

*Ed. 3 – October 2020*



**Health&Safety**

**RAYS SPA**

via Francesco Crispi - 60027 Osimo (AN)

Tel. 071 2868935 - Fax 071 2868911 - [www.rays.it](http://www.rays.it) - [info@rays.it](mailto:info@rays.it)