



Break the inside water bubble, by squeezing the bag with both hands.
Premere con entrambe le mani verso il centro fino ad ottenere la rottura della bolla interna.
Casser la bulle à l'intérieur du sachet en pressant avec force des deux mains.
Presionar con las dos manos hacia el centro hasta obtener la rotura de la burbuja interna.



Then shake it in order to mix the contents.
Agitare il sacchetto per mescolarne il contenuto.
frotte agiter le sachet pour mélanger le contenu.
Agitar la bolsa para mezclar el contenido.



Apply on the interested part.
Applicare sulla parte interessata.
Appliquer sur la partie intéressée.
Aplicar sobre la parte interesada.



Instant cold compress Ghiaccio istantaneo in busta PE

- Pochette de froid instantané
- Hielo instantáneo (frío)
- Sofort-Kälte-Kompresse (Kalt)
- Gêlo instantâneo (frio)
- Στιγμιαίος πάγος (κρύος)
- تلج فوري (بارد)

INSTRUCTIONS FOR USE: break the inside water bubble, by squeezing the bag with both hands. Then shake it in order to mix the contents. Apply on the interested part.

INDICATIONS: haematomas, sprains, injuries, hyperthermia and whenever cold application is required. **CAUTION:** disposable product for external use only. Do not break the bag or cause the outflow of the contents. Keep away from heat sources and inflammable substances. Keep out of children's reach. Use it at room temperature. The contents stain. If ingested the contents are noxious. In that case consult a doctor immediately. Do not apply on injured skin. Before using on babies, children, aged people and people affected by diabetes, cardiovascular and circulation disorders or skin insensibility (syndrome of Raynaud) consult a doctor. If in doubt on the treatment to carry out or if the symptomatology persists, consult your doctor.

CONTENT: mixture of nitrogenous salts (as required by EC Regulation 1907/2006 and as amended) and water.

REF **34111** 14x18 cm - PE bag

 **Fabbricante/Manufacturer:**
Gima S.p.A - Via Marconi, 1
20060 Gessate (MI) - Italy
Made in Italy




GIMA
www.gimaitaly.com