GARANZIA

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WARRANTY

This toilet safety rail has **2-year** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. **The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.** In case you need to get your bench repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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Acquistato in data / Purchasing date _____

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TOILET SAFETY RAIL INSTRUCTION MANUAL

Moretti S.p.A.
Via Bruxelles 3 - Melegnano
52022 Cavriglia (Arezzo)
Telefono +39 055 96 21 11

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I Class Medical Device
according to EEC directive 93/42

Code RS910

Thank you for purchasing the toilet safety rail **MOPEDIA** by MORETTI, which has been manufactured according to the highest quality and safety standards and in accordance with the European Directive on Medical devices (93/42/EEC). Realised in anodized aluminium.

Assembly instructions

Follow the drawing to assembly by screws the front bar (A) and the rear bar (B).
Make sure all screws are tightened enough .

To adjust height

press the stopper and slide the cane to the desired height. Put the stopper and make sure it is fully protruded through the desired hole. Check all four legs have the same height.

Cleaning

Mopedia safety toilet rail is designed for minimum maintenance and requires the occasional wipe over with a soft cloth. Use general purpose neutral detergent and warm water.
Don't deep the parts in water. Drain the seat tubes after use and dry all parts thoroughly.

Warning

- This product is recommended for people weighting kg.100 maximum.
The manufacturer and distributor decline all responsibility for any consequences resulting from an incorrect use of this product.
- Check that the product has been delivered in perfect condition before you use it.
 - In case of doubts about the use, adjustment or maintenance of the product, please contact your dealer.
 - Worn, damaged or missing components must be replaced.
 - Do not store the product outdoors.
 - Periodically check the stability of the product, screws are properly tightened and the rubber caps end in good conditions.
 - Avoid to use toilet satey rail on wet surfaces.