MEDISANA®

Ultraschall-Inhalationsgerät USC Ultrasonic Nebulizer USC

Inhalateur à ultrasons USC
Nebulizzatore ultrasonico USC
Nebulizador por ultrasonidos USC
Nebulizador por ultra-sons USC
Ultrasoon inhalatieapparaat USC
Ultraääni-inhalaattori USC
Ultraljuds-inhalationsapparat USC
Εισπνευστήρας υπερήχων USC

www.elettromedicali.it

Art. 54100

CE 0297

Gebrauchsanweisung
Manual
Mode d'emploi
Istruzioni per l'uso
Instrucciones de manejo

Bitte sorgfältig lesen!
Please read carefully!
A lire attentivement s.v.p.!
Da leggere con attenzione!
¡Por favor lea con cuidado!

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1.1 Thank you

Thank you for your confidence in us and congratulations on your purchase!

By purchasing the ultrasonic nebulizer **USC** you have acquired a quality product by **MEDISANA**. To ensure that you are able to obtain the results you are looking for and can enjoy using your **MEDISANA** nebulizer **USC** for a long time to come, we recommend that you read with care the following information about using and caring for the device.

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

1.2 Important

- Use the device only according to its intended purpose as specified in the instruction manual.
- Only use genuine accessories and spare parts made by the manufacturer.
- Avoid contact with pointed or sharp objects. The warranty will be invalidated if used for purposes other than those for which it is intended.
- Do not attempt to repair the device yourself in the event of a malfunction. Repairs should only be carried out by authorised service centres.
- Supervision should always be in place if the device is used by, on or near children or people who are handicapped and can only make limited use of the device. Medical devices are not toys!
- Hold the device as upright as possible during use.
- Do not operate the device in the vicinity of anaesthetic mixtures that are inflammable in air or oxygen.
- Do not operate the device in the vicinity of high-frequency electromagnetic transmitters.
- Never use the device whilst taking a bath or shower.
- When cleaning, do not hold the whole device or the nebulizer chamber under running water. Take care to ensure that no water gets inside the device.
- Do not leave the device outside or in a humid environment. The device is not protected against the ingress of liquids.
- If you are not going to use the device for a lengthy period, take out the batteries to prevent them leaking.
- Before beginning therapy with the device, consult your doctor regarding dosage, as well as the length and frequency of use.

1.2 Important

- **The nebulizer is not suitable for using in respiration systems for anaesthesia or for artificial respiration systems.**
- Consult your family doctor before using the device if you have diabetes or other illnesses.
- Pregnant women should observe the necessary precautions and their personal circumstances; consult your doctor if necessary.
- Do not use the device if the application will come into contact with bruises, burns, inflammations, rashes, wounds or sensitive areas.
- The treatment should be pleasant. Stop using the device and consult your doctor if you feel pain or using the device is unpleasant.

2.1 Items supplied and packaging

Please ensure that the package contents are complete.

These are as follows:

- 1 Nebulizer **USC**
with nebulizer unit, active ingredient tank and cover
- 1 Mouthpiece
- 1 Joining piece with mask
- 1 Adults' mask
- 1 Children's mask
- 4 Batteries 1,5 V, LR6 AA
- 1 Storage bag
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

To avoid risk of suffocation, keep packaging film away from children!

2.2 Important information about inhalation therapy

Living = breathing / Breathing = living

Often, diseases of the respiratory system or the lung are lengthy, painful and life restricting. Timely, effective therapies such as inhalation therapy with aerosols can be of great help and assistance.

A gas + particle mixture is defined as an aerosol; mist is an example of a natural aerosol. Aerosol therapy delivers medicines in an "air transported" form (e.g. nebulized) which are inhaled. The great advantage of this therapy (unlike taking tablets for

example) is that the medicines are administered at the precise point where they will have greatest effect (via the oro-nasal-pharyngeal cavity into the lung) without passing through other organs:

1. The effect is noticeably rapid.
2. There is no (significant) effect or side effect on other organs.
3. Small quantities of medicines are used.

2.3

Special features of the MEDISANA USC

Ultrasound technology

An aerosol is generated in the ultrasonic nebulizer due to the vibrations of a piezoelectric crystal.

Thus, electric energy is transformed into mechanical oscillations which create droplets. The optimum diameter of the droplets is 1 - 5 μm (according to "Guidelines for the aerosol therapy of lung diseases", Lungenliga Schweiz). The droplets can only penetrate into the narrow branches of the upper and lower airways if they have a small diameter thus transporting and depositing the medicine where it is intended to be most effective.

Inhalant

Important information when using preparations/medications

ATTENTION

- Consult your doctor before using any medicinal products or medicines to ensure that they are suitable for your needs or alternatively only use the medicinal products/medicine prescribed for you by your doctor.
- The MEDISANA USN ultrasonic inhaler is suitable for use with all common aerosol products (aerosols) with the exception of
 - Products with solid constituents (suspensions)
 - Viscous solutions (high viscosity)
 - Volatile oils
- Do not use any volatile oils since this may lead to damage to the nebulizer diaphragm.
- Use only water-soluble products/medicines or those which have an alcohol base.
- Use only isotonic saline solution to dilute the inhalant; otherwise bronchial spasms may occur in certain circumstances.
- Do not use oil-based inhalants.

Accessories

There is no provision for combining with other devices.

Use only the original accessories with this device since only then can safe operation be guaranteed.

Use a separate accessory set for each user to avoid cross-infection or the transmission of harmful substances.

Leakage safety

When pouring the inhalant into the active ingredient tank ②, only fill it to the maximum mark (8 ml). The recommended filling quantity is between 2 and 8 ml.

The **MEDISANA USC** is leakage proof. Do not, however, shake the device when the active ingredient tank is full; there is a chance that liquids could get into the electronics and destroy the device.

Automatic limit stop

The device is equipped with an automatic limit stop: the device switches off automatically after a short time if the inhalant is used down to a very low residual quantity and there is hardly any or no liquid in the active ingredient tank.

Do not operate the device if the active ingredient tank ② is empty.

The device also switches off automatically if the inclination of 45 ° is exceeded in any direction.

3.1 Assambly

1. Place the nebulizer unit **3** on the attachment on the active ingredient tank **2** and lock it by turning it clockwise until the gold contact electrodes face down.
2. Now place this combined module on the base of the device, so that the white catches on the left and right click into place.
3. Put on the cover **1** so that it too audibly clicks into place on all four catches.
4. Put in the batteries.

3.2 Inserting / removing batteries

Open the battery compartment **6** on the underside of the unit by exerting slight pressure whilst sliding it simultaneously in the direction of the arrow. Insert the four batteries (type AA LR6) supplied. Remember to insert the batteries as indicated in the illustration on the base of the device.. Replace the lid, pressing it until it snaps into place.

If the orange battery indicator **4** lights up when the device is switched on, replace the batteries.

BATTERY SAFETY INFORMATION

- **Keep away from children!**
- **Do not recharge!**
- **Do not short-circuit!**
- **Do not throw into fire!**
- **Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop!**

3.3 Service life

With new alkaline batteries, the device can run for approximately 100 minutes. This is sufficient for 5 - 8 inhalation sessions with salt solution (depending on the dosage and length of use).

As a rule, the inhalation phase using the **MEDISANA USC** is 8 - 10 minutes if the active ingredient tank is filled to its maximum of 8 ml due to the ultrasound nebulizer's high performance. However, each inhalation session may not last more than 15 minutes.

3.4 Area of application

Possible areas of application for inhalation therapy are acute diseases of the airways such as.:

Colds

Inflammations of the paranasal sinuses

or chronic diseases such as.:

Bronchial asthma

Bronchitis

IMPORTANT

Before starting therapy, discuss the length, dosage and frequency of use with your doctor. The method of use should be appropriate to the aim of the therapy (e.g. lung treatment = use of mouthpiece and treatment of the nasal-pharyngeal area = use of mask). Consult your doctor about this aspect as well.

3.5 What can the device be filled with?

Use only water-soluble preparations/medicines or product that are alcohol based. All common inhalation solutions for ultrasound inhalation devices can be used. Salt solution is often recommended. Do not use oil-based inhalants. Ask your doctor which preparations are suitable for you.

More detailed information on the topics of "Aerosols", "Inhalation" or "Breathing Therapy" can be found in specialist books, or obtained from study groups and associations, or from your doctor.

3.6 Preparation for inhalation

Before operating for the first time or if the device has not been used for some time, clean and disinfect the nebulizer unit as well as the mouthpiece and mask including the connecting piece (also refer to "Cleaning and maintenance").

Remove the cover **1** from the device. Open the active ingredient tank **2** and fill it with inhalant (maximum capacity 8 ml = 8 ccm). Please observe the dosage and instructions for use on the information leaflet enclosed with the medicine you are using. Close the active ingredient tank lid again and place the cover **1** on the device. Make sure the cover is correctly seated and sealed, i.e. that it is latched in place. Then plug the mouthpiece **9** or the connecting piece **7** and mask **8** into the attachment port of the nebulizer unit **3**.

3.7

Inhalation using the mouthpiece

1. Sit as upright as possible.
2. Enclose the mouthpiece ⑨ completely with your lips.
3. Switch on the device by pressing the ON/OFF key ⑤. The equipment-on indicator lamp (green, ④) lights up. The aerosol warms up slightly during operation.
4. Now breathe in slowly and deeply through your mouth and out through your nose. You can improve the effectiveness of the therapy, if you hold your breath for a short time after breathing in. However, this is not necessary for general success of the therapy. It is only important that you keep quiet and relaxed while breathing in and out regularly. Do not inhale too fast. If you need a break, stop using the device for a short time and take the mouthpiece out of your mouth. Then put it into your mouth again (lips enclosing it tightly) and breathe slowly in and out again.

ATTENTION

Hold the device as upright as possible. A slight tilt does not affect its use (leakage proof). However, take care during inhalation not to tilt the device more than 45 ° in any direction.

5. An inhalation application should not exceed 15 minutes.
6. After terminating the inhalation, switch off the device by pressing the ON/OFF key ⑤ even if you have not used up all the inhalant. The equipment-on indicator lamp (green, ④) goes out.
7. The device switches off automatically after a short time (automatic limit stop) if the inhalant in the nebulizer is used up. For technical reasons a small amount remains in the nebulizer unit.
8. Pour out the rest of the inhalant. Do not use it again.
9. Clean the device immediately after each use. (Also refer to "Cleaning and maintenance").

Once you have used the mouthpiece for a certain time, two or three drops of inhalant solution may gather in the attachment pipe. However, this does not affect the operation and function of the device. Wipe off the drops with a soft cloth after use.

3.8 Inhalation using the mask

Alternatively, two inhalation masks **8** (for children and adults) and a connecting piece **7** are supplied with the device. They can be used for inhaling aerosol through the nose. This method is particularly suitable for children and elderly people.

1. Sit as upright as possible.
2. Plug the mask **8** with the connecting piece **7** onto the nebulizer unit **3**.
3. Hold the mask pressed lightly over your nose. Make sure that the mask seals around the nose without being too tight.
4. Switch on the device by pressing the ON/OFF key **5**. The equipment-on indicator lamp (green, **4**) lights up. The aerosol warms up slightly during operation.
5. Now breathe in and out slowly and deeply through your nose. You can improve the effectiveness of the therapy if you hold your breath for a short time after breathing in. However, this is not necessary for general success of the therapy. It is only important that you keep quiet and relaxed while breathing in and out regularly. Do not inhale too fast. If you need a break, stop using the device for a short time.

ATTENTION

Hold the device as upright as possible. A slight tilt does not affect its use (leakage proof). However, take care during inhalation not to tilt the device more than 45 ° in any direction.

6. An inhalation application should not exceed 15 minutes.
7. After terminating the inhalation, switch off the device by pressing the ON/OFF key **5** even if you have not used up all the inhalant. The equipment-on indicator lamp (green, **4**) goes out.
8. The device switches off automatically after a short time (automatic limit stop) if the inhalant in the nebulizer is used up. For technical reasons a small amount remains in the nebulizer unit.
9. Pour out the rest of the inhalant. Do not use it again.
10. Clean the device immediately after each use. (Also refer to "Cleaning and maintenance").

 **NOTE**

A larger amount of aerosol remains on the mucosa of the nasal pharynx when using the mask than when using the mouthpiece.

4.1 Extra Tip from MEDISANA

The **MEDISANA USC** with its storage bag means you can take your ultrasound inhaler with you safely and cleanly when travelling. Due to its low noise level, you can easily use the inhaler without disturbing your neighbours. Especially when treating children, it has been noted that the low-noise function leads to a higher level of acceptance of use since it is considered less disturbing and frightening.

 **NOTE**

If you have several inhaler users in one household, and thus several accessory sets, we recommend that they are clearly marked, e.g. with a permanent ink pen, to prevent any mix up.

4.2 Cleaning and maintenance

 **IMPORTANT**

Carry out the cleaning instructions carefully directly after each use as described below.

- Remove the batteries before cleaning the device.
- Take the attachments, cover, active ingredient tank and nebulizer unit off the device.
- When cleaning, do not hold the whole device or the nebulizer unit under running water. Take care to ensure that no water gets inside the device.
- Clean the device's casing with a lightly moistened cloth that has been soaked in mild soapy water.
- Clean the cover in the same way.
- Normally, you can clean the used attachments (mouthpiece, mask, connection piece) under the tap.
- This cleaning routine should also be carried out if the device has not been used for a longish period of time and prior to using for the first time.

- In the case of infectious illnesses, the device must be disinfected every day or all attachments must be sterilised (it is recommended that this measure is performed regularly either in the morning or the evening at a fixed time). You can also use alcohol or other common alcohol-based disinfectant sprays. Make sure that no disinfectant residues are left behind on the attachments to ensure safe inhalation when next used.
- Allow all parts to dry separately and uncovered on a cloth. Do not push the extension parts together whilst they are still wet.
- Clean the active ingredient tank and the nebulizer unit after each use. Put 8 ml of ethyl alcohol in the active ingredient tank. When doing this, do not remove the nebulizer unit from the medicine tank. Close the active ingredient tank. Leave the alcohol in the tank for at least 10 minutes. For more effective disinfection, gently shake the unit from time to time. After this treatment, tip the ethyl alcohol out of the tank.
- Repeat the procedure, this time using water.
- Now disconnect the nebulizer unit from the active ingredient tank. To do this, turn the nebulizer unit anticlockwise and pull it off the tank attachment. Finally, clean both parts again in fresh water.
- Always make sure the nebulizer unit and active ingredient tank are properly dried out.
- If there is medicine solution or dirt on the gold contacts between the device and the nebulizer unit, clean them with a cotton bud soaked in ethyl alcohol.

4.3 Disposal

This product must not be disposed together with the domestic waste.

All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.

Consult your municipal authority or your dealer for information about disposal.

4.4**Accessories for
MEDISANA USC**

Ask at your specialist
dealer or call our hotline
directly.

Mask set (2 masks + adapter) for USC

Article number 54101 EAN number 40 15588 54101 8

Mouthpiece for USC

Article number 54102 EAN number 40 15588 54102 5

Covering cap for USC

Article number 54103 EAN number 40 15588 54103 2

Nebulizer chamber for USC

Article number 54104 EAN number 40 15588 54104 9

Use only the original accessories with this device since only then
can safe operation be guaranteed.

4.5**Specifications**

Name and model	: MEDISANA Ultrasonic Nebulizer USC
Power supply	: 6 V= , 4 x 1,5 V batteries AA LR6
Ultrasound frequency	: 100 kHz
Nebulizing output	: more than 0.25 ml per minute with salt solution
Average particle size	: 4,8 µm
Filling volume inhalant	: max. 8 ml = 8 ccm
Residual volume in medicine tank	: ca. 0,2 ml
Dimensions	: approx. 142 x 75 mm
Weight	: approx. 300 g
Operating conditions	: 5 °C to 40 °C, less than 80 % relative humidity, non-condensing
Storage conditions	: -10 °C to +45 °C, less than 80 % relative humidity, non-condensing
Classification	: Device classified type BF  Classified according to MDD: IIa
Article number	: 54100
EAN number	: 4015588 54100 1

CE 0297

**We reserve the right to make technical and design
changes in the course of continuous product
improvement.**

Electromagnetic compatibility:

The device complies with the EN 60601-1-2 standard for
electromagnetic compatibility.

The following standards were applied and met:

EN 60601-1-2 (CISPR 11: 1997+ A1: 1999 + A2: 2002)

EN 60601-1-2 (IEC 61000-4-2: 1995 + A1: 1998 + A2: 2000)

EN 60601-1-2 (IEC 61000-4-3: 2002 + A1: 2002)

5.1 Warranty and Repair Conditions

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

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