GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 - fax 199 400 403
Export: tel. +39 02 953854209/221/225 fax +39 08 95380056
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

PETTINE ANTI PIDOCCHI **LICE COMB** **PEIGNE ANTI-POUX** **PEINE ANTIPIOJOS**

Manuale utente
User manual
Notice d'utilisation
Manual de uso

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ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

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ENGLISH

Eliminate the lice and nit completely without any harm.

UTILIZATION:

- The lice are the small insects which host in hair and scalpel. The lice lay the minuscule eggs which are smaller than 0.5mm, called as the nit.
- The nits which are protected with a white chitin cover attach strongly to the hair.
- The long, delicately closed teeth of the lice comb are especially designed for detaching the lice and their nits. The round form of the touching point respects the growth direction of the hair without harming the texture of hair. The curve arrangement of the edge ensures the precision and smoothness of the comb, as respecting the curve of your scruff.

ADVANTAGE:

- Natural, mechanical, totally ecologic method; ideal for the allergic user and the children.
- Completely useful than other treatment with an efficiency of close to 100%.

INSTRUCTION:

- Disentangle the hair before the usage for facilitating the passage of the special teeth. Your hair should be dry.
- Little by little slowly comb your hair from the root to the end, pay attention to every spot of host for the lice or nit.
- The built in magnifier and the cleaning brush on the cover cap help mousing out the smallest lice and verify a great carding.
- Repeat the operation several times if necessary, notably during the period of infection or for the prevention.

CLEANING AND DISINFECTION:

Use the brush in order to carefully clean the teeth of the lice comb after the usage. It's recommended to disinfect the metallic part with 70% alcohol after each use.

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