# GIMA

## GIMETTE 50

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**

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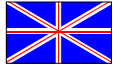
I = main power switch on

0 = main power switch off

Dangerous temperature surface indication

Grounding: 

## TECHNICAL DATA

Absorbed power	1200 Watt
Voltage/Frequency	230 Volt - 50 Hertz
Fuse	2x8 A
Timer	Mechanical type 0 ÷ 120 min.+ manual position
Thermometer	High precision with capillary and bulb
Thermostat	Automatic with capillary tube having liquid expansion probe
2nd safety thermostat	Certified bimetallic thermostat
Overall size (w.d.h.)	cm. 60,5 x 46 x 48 approx.
Internal size (w.d.h.)	cm. 40,5 x 34 x 35 approx.
Weight	Kg. 24,5 approx.

## DESCRIPTION

The sterilization chamber is entirely made of bright stainless steel resistant to heat and corrosion. It is fully insulated with fibra glass to avoid the overheating of external side. The door closes sterilization chamber hermetically by means of a silicon gum gasket resistant to heat. The sterilizer has five anodized aluminium trays (four extractable trays and one as a bottom surface) with holes for warm air circulation. The guides on which the trays run are separated from the sides to allow excellent warm air circulation (to obtain maximum heat distribution it is recommended not to use the bottom surface).

The sterilizer has two stainless steel armoured electrical certified resistances 600 Watt each; they are characterized by high efficiency and little inertia. The automatic control of temperature is obtained with a capillary bulb automatic thermostat for auto-adjustment up to 200°C.

If the temperature exceeds 200°C, a second safety bimetallic thermostat takes control.

The working of the second thermostat causes the alarm red pilot lamp lighting.

The thermostat starting time is regulated either with a timer (up to 120 min.) or manually.

The setting of the required temperature and time, is obtained with the thermostat and timer knobs.

The internal temperature of the sterilization chamber is indicated by an high precision thermometer with capillary bulb.

The device is also provided with a bright main power switch, with green, orange and red lights for working, heating and alarm respectively, and protection fuses.

The sterilizer is delivered with certified supply cable, four anodized aluminium trays and one holdertrays pliers.

## INSTRUCTIONS

### Carriage

The sterilizer is shipped in an adequate package that requires the common handling cautions for fragile materials, as shown on the package itself. Use only the original package if the equipment must be returned to the manufacturer. To carry the sterilizer without package, keep it in a horizontal position and use the base as holding point.

### Installation - start up

Remove the equipment from the package handling with care. Check the power supply system is correctly grounded. Place the sterilizer on a level surface so that air can freely circulate under the equipment bottom side. Keep the back side of the sterilizer at least 10cm. away from walls to ensure an adequate ventilation. Insert the trays in their housing.

Insert the safety plug of the power supply cable, firstly into the equipment socket and then into the electric system, after being sure the voltage corresponds to the one selected on the device. Press the green main power switch position "I". Check if the switch light is on.

Note: a typical smell of heated ceramic fibre may be present at the first power on and remain for a short period of time.

### Instructions to use

- Press the main power switch position "I".
- Turn the thermostat knob (nr.4) positioning the index on the required temperature.
- Turn the timer knob (nr.5) clockwise, positioning the index on the required time, up to 120 minutes.
- Check the lighting of the orange light. The orange light (nr.1) (whose lighting points out the working of the resistance) will have some automatic clicks because of heat distribution into the sterilization chamber, before stabilizing on the required temperature.
- To disconnect the timer and to make the sterilization work for an indefinite time, turn the timer knob anticlockwise, on the "Manual" position.

For a correct use do not fill the trays up excessively; it will allow a good circulation of hot air. When the required temperature is reached, the sterilization times vary from min.15-20 minutes up to max. 45-60 minutes with temperature 150÷180°C max. according to the material. Do not keep the thermostat highest temperature for a long time, in order not to risk of spoiling the sterilized material.

The lighting up of the red light (nr.3) “Alarm” indicates the temperature is going over 230°C and the second safety thermostat is working limiting the temperature.

Let the service or the specialized technician verify the efficiency of the regulator thermostat.

### **Maintenance and repair**

Always check the power is unplugged before maintenance is carried out. The sterilizer needs no particular maintenance, provided the adequate environment are satisfied, except for a regular cleaning of its external parts.

It is recommended to clean the equipment with a soft cloth and water. Do not use acids or solvents. Regularly check if the power cable is in good conditions. It is possible to replace the safety fuse pulling out the fuse block at the rear under the socket. Please always follow the characteristics stated in the technical data sheet. If the sterilizer doesn't work properly, contact the manufacturer or the authorized service. If any repairs are needed, please return the equipment directly to the manufacturer or to the local distributor using only the original package. However it is possible to contact specialized technicians, who must work in accordance to general safety and accident prevention rules of the country in which the equipment is installed.

### **Out of order**

No particular cautions.

## **SAFETY CAUTIONS**

This equipment must be used only by authorized personnel thoroughly familiar with the correct use.

The sterilizer must be used only for its designed purpose:

*to sterilizer surgical instruments or tools in the medical and paramedical field. The sterilizer must not be used for all treatments concerning contact lenses (Rule 15 - All.9 - Comma 1).*

It is necessary to take all precautions to avoid unauthorized use.

Do not modify, change or put out of order any electrical or thermal safety protection of the equipment.

When the sterilizer is operating, pay attention to the people near the equipment.

The user must not attempt to carry out any repair unless authorized.

Do not clean or repair the sterilizer while it is connected to an electrical system and be sure the sterilization chamber is at room temperature. Regularly check if the power cable is damaged.

When authorized technicians decide to replace spare parts, they must use original replacements and moreover they must adhere to the manufacturer's information and technical characteristics. In case of any doubt about safety rules, please contact the manufacturer or local distributor before using the equipment.

However use the sterilizer in accordance with the general safety and accident prevention rules of the country in which the equipment is installed.

### **ATTENTION!**

*Avoid to use the sterilizer in an environment where the explosive gases may be present.*

**GIMA**

*Place the sterilizer on a level surface so that air can freely circulate under the equipment bottom side.*

*Keep the sterilizer at least 10cm. away from walls to ensure an adequate ventilation.*

*Keep the door closed when the sterilization process is running.*

*Do not place any object on the upper part of the equipment.*

*To extract the trays and fit in, use the appropriate holdertray plier.*

*Do not clean, move or repair the sterilizer while it is connected to an electrical system and the sterilization chamber is still hot.*

**ATTENTION!**

*when the sterilizer is operating do not put hands in the sterilization chamber and do not touch the trays.*

**MANUFACTURER'S RESPONSIBILITY**

The sterilizer must be used only for its designed purpose.

Improper use of the equipment is to the user's own risk and sole responsibility.

**GUARANTEE**

Thank you for having acquired our product.

This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, doppler probes, SpO<sub>2</sub> sensors, lamps, batteries, electrodes, handpieces, washers and packings, resistances, etc.

No compensation will be given for loss of use of the product.

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
Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered.

Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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