GARANZIA

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WARRANTY

The product has 2-YEAR warranty from purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge. The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals. In case you need to get the product repaired and you fulfil the conditions of warranty, send it to the nearest retailer together with the purchase receipt.

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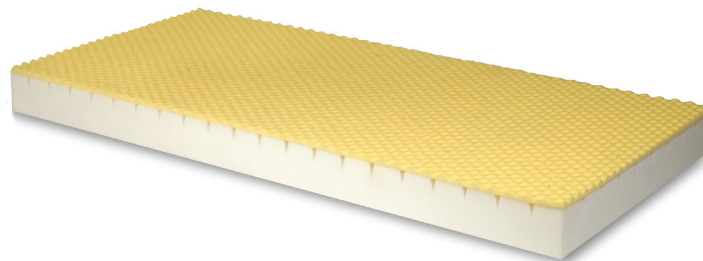
Acquistato in data / Purchasing date _____

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**POLYURETHANE TWO-COMPONENTS MATTRESS,
ONE SECTION PERFORATED AND RUSTICATED
USER'S MANUAL**

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Medical Device Class I
Directive 93/42 EEC

Code ST728

Technical features:

Thank you for purchasing the polyurethane 2 components mattress **"LEVITAS"** by Moretti. The base is made of expanded polyurethane (density 30kg m³). The upper part is perforated and rusticated and it's made of viscoelastic foam (low resilience, density 50 kg m³). Without cover. (**The cover is optional: code ST760 - ST761**)

- **ST728** _____ cm 190X85X14h - 7,2kg
Max loading weight 120kg
Made in one section

Cleaning:

"LEVITAS" mattress in polyurethane or viscoelastic material can be autoclaved 100° for a maximum of 10 minutes. Alternatively you can use a high pressure water jet (like a pressure water machine) with a neutral soap. The ideal solution could be an air centrifugation system, to allow the used water to weep away completely from the mattress. Finally dry for 48 hours far from sun rays and warmth source.

Warning:

We suggest the use of the mattress one person at a time. Use and keep the mattress in a horizontal position, away from fire and from heat source. The device has to be sanitized before the use of another patient. There aren't known side effects.