seca

seca 220

Deutsch	Seite 3
English	Page 6
Français	Page 9
Italiano	Pagina 12
Español	Página 15
Dansk	Side 18
Svenska	Sida 21
Norsk	Side 24
Suomi	Sivu 27
Nederlands	Pagina 30
Português	Página 33
Ελληνικά	Σελίδα 36
日本語	39

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

- Set measuring slide to a value, e.g. 150 cm.

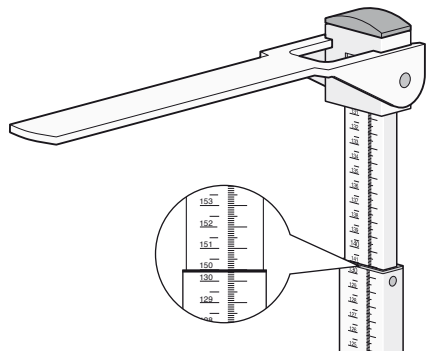
- Using another measuring rod, measure the difference from the platform (models 709, 710, 709f, 710f: **from the floor**) to the measuring slide and note the difference.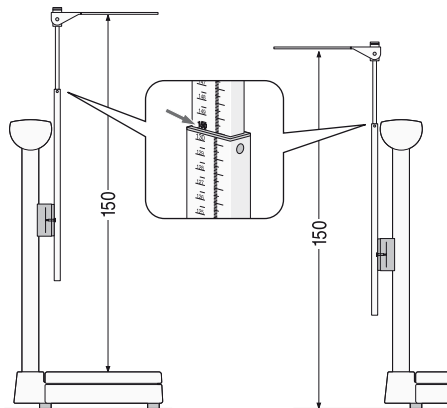

Model 220

- Undo the screw on the measuring rod somewhat, move the slide and measuring rod up or down according to the difference and tighten it back up.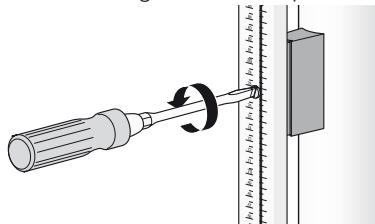

- Check height again and correct if necessary.

The measuring rod is also easily readjusted in this manner if the scale is moved and realigned using the adjustable feet.

Note:

For fine-adjustment of the measuring rod, it is essential to measure from the floor/platform to the measuring slide. Do not measure from the floor/platform to the relevant height on the scale. The scale is displaced a few centimetres to compensate for the difference in height between the read-off mark and the measuring slide.


Operation

How to measure properly...


- Engage measuring slide in the horizontal position for measuring.
- Push measuring slide upwards according to the height of the person to be measured.

For measuring from 130.5 cm to 200 cm (from 51.3 inch to 78.7 inch):


- read-off mark (1)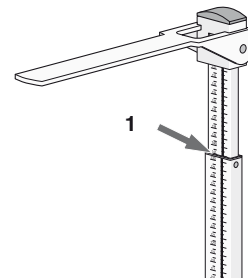

for measuring below 130.5 cm (51.3 inch):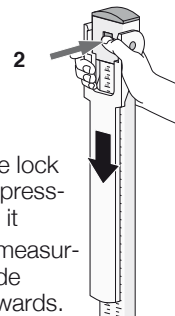

- release lock (2) by pressing on it
- push measuring slide downwards.
- read-off mark (3).

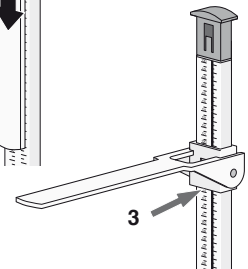- The person to be measured positions him or herself under the measuring slide. The back and head must be straight.

- The measuring slide is pushed onto the head to that the measuring slide abuts without bending.
- Read off body height at the read-off mark.
If the above-mentioned instructions are followed, accuracy greater than ± 5 mm can be achieved.

Cleaning

Clean the measuring rod as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Technical data

Measurement range:	60-200 cm / 24-78 inch
Graduations:	1mm /1/8 inch
Accuracy	greater than ± 5 mm
Dimensions (W x H x D):	55 x 784 x 34 mm
Weight:	ca.700 g
Temperature range:	+10 °C up to +40 °C
Medical product in accordance with Directive 93/42/EEC:	Class I with measuring function

Disposal

Should the measuring rod no longer be able to be used, the responsible waste disposal association will be glad to give you information about the measures necessary to dispose of it properly.

Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be honoured only if the complete original packaging was used for any transport and the merchandise secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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