
# Kopfleuchte HEINE MD 1000® F.O.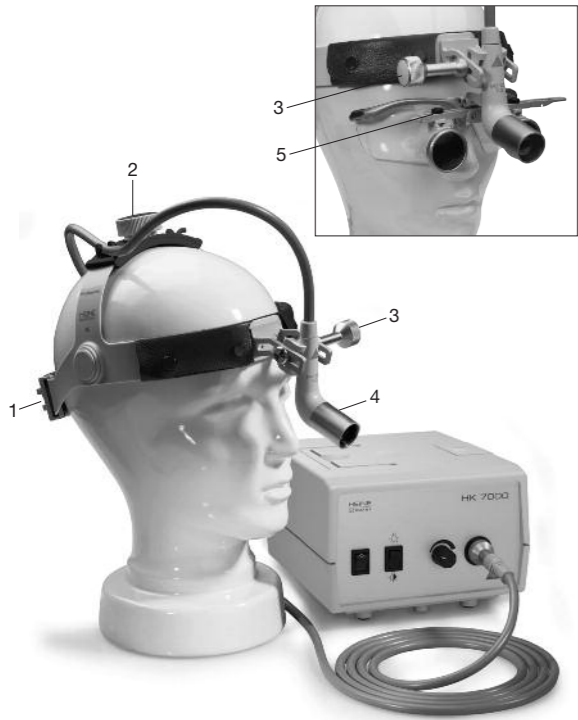

CE 93/42/EWG/CEE/EEC

## HEINE

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49(0)81 52/38-0 · Fax +49(0)81 52/38-202  
E-mail: info@heine.com · www.heine.com  
med 3801 2/12.08

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DEUTSCH

### Kopfleuchte HEINE MD 1000® F.O.

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ENGLISH

### HEINE MD 1000® F.O. Headlight

Before using the HEINE MD 1000® F.O. headlight read these instructions carefully and keep them in a safe place for future reference.

#### Safety informations

**Range of applications:** The HEINE MD 1000® headlight is used for examination and during treatment.

**Safety in use:** Use only light sources which are approved for use in a medical environment. Do not shine the light directly into the patient's eyes.

#### Adjusting the beam direction

- Plug in the fiber optics cable to your light source and switch on.
- Place the instrument on your head and adjust for width (1) and height (2) to that it fits comfortably.
- Loosen the fixing screw (3) to adjust the light head up and down and backwards or forwards.
- After adjustment, re-tighten the screw (3).

#### Selecting viewing angle and field of view

##### Without HEINE S-Guard®

Loosen the fixing screw (3) to adjust the light head up and down and backwards or forwards. After adjustment, re-tighten the screw (3).

##### With HEINE S-Guard®

- Before putting on the loupes, select your inter-pupillary distance (PD) or an average value on the scale. Unscrew the retaining screw (5) and slide the eyepieces to one side symmetrically.
- Unscrew the fixing screw (3) and locate the nosepad on the bridge of the nose.
- Adjust the eyepieces so that they nearly touch the protective lenses.
- Shine the 3S LED onto an object at your normal working distance and adjust the eyepieces so that the object is centred in both eyepieces. After adjustment, re-tighten the screw (3).

The loupes are correctly adjusted when you see a clearly-defined, round, shadow-free field of view. Please consult the instruction leaflet for HEINE HR and HRP binocular loupes.

The illuminated field of view can be selected by means of the adjusting ring (4).

The adjusting ring can be fitted with a sterilisable sleeve (see accessories).

**Fitting HEINE binocular loupes and a video camera:** Use the appropriate adaptor, which is available as an accessory.

#### Supporting the fiber optics cable

The F.O. cable includes adjustable clips which are used to attach the cable to a gown or similar.

#### Cleaning

Use a micro-fiber cloth to clean the lens. The instrument housing can be cleaned with a soft cloth moistened with alcohol or a non-aggressive disinfectant solution. Do not soak or spray with disinfectant. The sterilisable sleeve can be autoclaved at 135 °C or sterilised with ethylene oxide.

Read the instructions for cleaning the HEINE i-View®.

#### Technical Specification

Weight:	ca. 230 g (cable supported by clips)
F.O. cable length:	300 cm
Diameter of light spot at 50 cm distance:	ca. 40 - 160 mm
Type of operation:	continuous

#### Spare Parts and Accessories

Adaptor for HEINE HR®-loupes	J-000.31.212
Adaptor for HEINE G- and K-loupes	J-000.31.213
HEINE MD 1000® F.O. video prism optics	J-000.31.208
Sterilisable sleeve	J-000.31.222
3 adjustable gown clips	J-000.31.223
HK 7000 fiber optics projector	Y-096.15.121
XENON 1000	Y-096.15.117

<b>Conversion set for HR Binocular Loupes</b>	<b>J-000.31.280</b>
<b>Retrofitting set Binocular Loupes</b>	
with HR Binocular Loupes 2,5x / 340	J-000.31.270
with HR Binocular Loupes 2,5x / 420	J-000.31.271
with HRP Binocular Loupes 3,5x / 420	J-000.31.272
with HRP Binocular Loupes 4x / 340	J-000.31.273
with HRP Binocular Loupes 6x / 340	J-000.31.274

## Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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