MultiEquipment. Ref F-335

Manual de instrucciones

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---

## Advertencia

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---

## Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

---

## EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2006/95/EC
2. Conditions set out in the Electromagnetic Compatibility Directive: 2004/108/EC

---

## Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

---

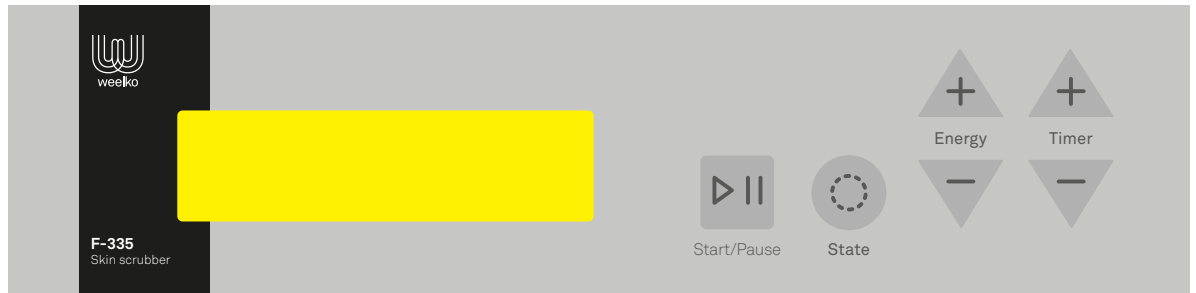
## Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

---

## Front panel

---

## Instructions of use

1. Connect the power supply via the rear panel.
2. Connect the accessories properly to the front/rear panel.
3. Turn on the “POWER” button.

---

## SKIN SCRUBBER

1. Place the bracelet around the customer’s wrist.
2. Apply the appropriate conductive product generously to the surface of the skin.
3. Select the desired wave type by pressing “STATE”, the desired output energy by pressing “ENERGY” and time of working by pressing “TIMER”. You can change setting anytime during working.
4. Press “START/PAUSE” button to start. Once there is circuit between the client and the scrubber, machine will start to work. The Scrubber sign on the screen will run when machine is working normally. Once there is separation between the scrubber and the client, machine will pause, and the Scrubber sign on the screen will stop running.

---

## Notes

A positive earth control system is in place while the machine is operating. This means that unless the machine can find an earth return, the system will stop the current output to the probe. However, once the probe comes back into contact with the customer (the earth circuit is found), the system will restart the output immediately. This is a safety feature and should not be mistaken for a system fault.

---

## Cautions

- The ones with the following diseases are required not to use it: heart disease, infectious disease, a person with metal in body due to treatment, a person with extremely sensitive physical structure.
- Do not keep the probe at the same position of the skin too long to avoid burning.
- In case the exclusive liquid is insufficient, supplement it.
- As the probe is fitted and unloaded, please do it in accordance with due order. Do not pull it off with force.
- As the power wire is disconnected from the back, pull it off from the socket. Do not pull its wire off.
- Do not hit the horn of the probe or damage the stainless steel surface to avoid skin scratch.

---

## **FAQ**

The probe suddenly stops emitting:

- If the probe is separated from the skin, it will stop emitting after 5 seconds. This is a safety feature and should not be mistaken for a system fault.
- Touch the skin with the probe and it will start emitting again after 5 seconds.

---

## **MAINTENANCE**

- Any oil on the probe must be removed immediately.
- After use, please clean the probe and keep it probe. Store it in the probe-protecting sleeve at all time.
- Store the probe carefully to avoid damaging it.

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