ERRATA CORRIGE

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
Mod. TABITHERMAE

IMPORTANT SAFEGUARDS


- Before using the unit for the first time, and periodically throughout its life, check the power supply cord for any damage; if you detect any damage, do not plug it in. Take the appliance immediately to an authorised FLAEM service centre or your dealer.
- Several parts of the device are small enough to be swallowed by children; therefore keep the device out of reach of children.
- Keep the cord away from heated surfaces.
- Do not handle the unit with wet hands. Do not use the device in humid environments (e.g. while taking a bath or shower). Do not immerse the device in water; should this accidentally occur, unplug the device immediately. Do not remove or touch the device immersed in water before pulling out the plug. Take the device immediately to an authorized service center or to your local dealer.
- Do not expose the device to high temperatures.
- Do not put the device near heat sources, direct sunlight, or in hot environments.
- Repairs must be performed by authorized personnel only. Any unauthorized repairs will void the warranty and may pose a safety hazard for the user.
- **WARNING: Do not modify this equipment without authorization of the manufacturer**

Pag.6 – THE STANDARD COMPONENTS AND ACCESSORIES OF THE UNIT INCLUDED:

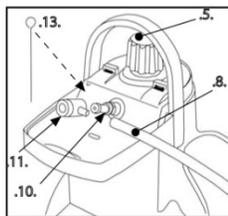
- .1. Handle
- .2. Rear cover
- .3. Front cover
- .4. Lighted on/off switch
- .5. Cap with safety valve
- .6. Used water collection tray
- .7. Funnel
- .8. Hose
- .9. Delivery outlet
- .9a. Diaphragm
- .10. Inhaler nozzle
- .11. Nozzle-hose connector
- .13. Nozzle cleaning pin
- .14. Water intake filter
- .15. Bottle cap
- .16. Bottle housing compartment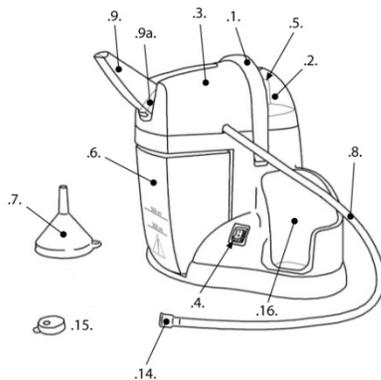

Pag.8– CLEANING

4.10 Remove the front cover (.3.) using a coin, and pushing upward as shown in the picture (fig.10)

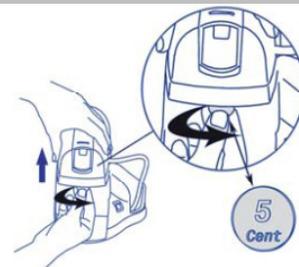4.14 Unscrew the nozzle (.10.) by turning it anticlockwise as shown in the picture (fig.12)

Pag. 10 – SPARE PARTS

Description	Code	Description	Code
- Inhaler Nozzle (complete with OR)	14706	- Bottle Cap	08968
- Nozzle-Hose connector	08467	- Funnel	08924
- Nozzle Cleaning Pin	08533	- Collection Tray	08465A0
- Water Intake Filter	08476	- Hose	02115

TECHNICAL DATA

Safety Certifications:

Type B applied parts are: none

SIMBOLOGIE

Warning: follow the operating instructions

Type B applied part

Do not use device while taking a bath or shower

TUV Certification
ref. EN 60601-1 3rd Ed.

ELECTROMAGNET COMPATIBILITY

This device has been designed to meet the current requirements for electromagnetic compatibility (EN 60 601-1-2:2007).- Electromedical devices require special care during installation and use, with respect to EMC requirements It is therefore required that they be installed and/or used according to the manufacturer's specification. Potential risk of electromagnetic interference with other devices, in particular with other devices for analysis and treatment I Radio and mobile or portable RF communications devices (mobile phones or wireless connections) may interfere with the operation of electromedical devices For additional information please contact Flaem at the toll free number 800-236622 and/or visit www.flaemnuova.it. The Manufacturer reserves the right to make technical and functional changes to the product without notice.