MATERASSINO AD ARIA A BOLLE
AIR BUBBLE MATTRESS
MATELAS PNEUMATIQUE À BULLES D'AIR
LUFTBLASEN-MATTE
COLCHÓN DE BURBUJAS DE AIRE
COLCHÃO DE AR OU BOLHAS
ΜΙΚΡΟ ΣΤΡΩΜΑ ΜΕ ΑΕΡΑ ΦΥΣΑΛΙΔΩΝ
MATERACYK NA PĘCZERZYKACH POWIETRZNYCH
LEVEGŐBUBORÉKOS MATRAC

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Brief

The BUBBLE MATTRESS consists of two parts: the mattress and the pump. The pump utilizes a small compressor to remain quiet and energy efficient. The control panel is simple and very user friendly. The mattress relieves pressure by sequentially deflating and inflating alternate air cells on approximately 6 minutes timed intervals. It is widely recognized that constant pressure to a bony prominence is the leading cause of skin breakdown. The continuous movement supplied by the unit lessens the areas of constant pressure and enhances circulation.

Operation

1. Place the mattress on the bed frame with the hose end at the foot section of the bed frame.
2. Using the integrated hooks, securely hang the pump on the bed end at the foot end or place on a smooth flat surface.
3. Connect the air hoses from the mattress to the pump.
4. Plug the pump into a wall outlet. Be sure the power cord is safely away from possible hazards.
5. Turn on the power switch on the control panel of the pump. The pump will begin to inflate the mattress.
6. After inflation, adjust the mattress using the dial on the pump.

Caution

1. Do not smoke on or near the pump.
 2. Keep the pump away from heated surfaces.
 3. Explosion risk if used in the presence of flammable anesthetics.
 4. Replace fuse as marked: T1A 250V.
- Power Input: AC 220V 50Hz, 0.1A

Product Specifications

DIMENSIONS	
MATTRESS (Length x Width x Height)	77"×35"×3" (195cm×88cm×7.5cm)
Extension Flaps	20"(50cm) For both sides
PUMP (Length x Width x Height)	10"×5"×4" (26cm×13cm×10cm)
WEIGHT OF PRODUCT	
MATTRESS WEIGHT	5 lbs (2.3kg)
PUMP WEIGHT	3 lbs (1.4 kg)
WEIGHT CAPACITY	
MAXIMUM PATIENT WEIGHT	298 lbs (135 kg)
ELECTRICAL	
OPERATING POWER	AC 220V 50Hz, 0.1A MAX
FUSES	T1A 250V
ENVIRONMENTAL CONDITIONS	
OPERATING CONDITIONS	
Ambient Temperature	10°C to 40°C (50°F to 104°F)
Relative Humidity	10% to 75%
STORAGE AND SHIPPING CONDITIONS	
Ambient Temperature	-18°C to +43°C (0°F to 110°F)
Relative Humidity	10% to 95%

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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