BILANCIA DIGITALE IN VETRO
GLASS DIGITAL SCALE
PÈSE-PERSONNES EN VERRE À AFFICHAGE NUMÉRIQUE
GLASPERSONENWAAGE
BÁSCULA DIGITAL DE CRISTAL
BALANÇA DIGITAL DE VIDRO
ΨΗΦΙΑΚΗ ΖΥΓΑΡΙΑ ΑΠΟ ΓΥΑΛΙ
SZKLANA WAGA CYFROWA

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FEATURES & SPECIFICATION

- Auto ON
- Auto OFF
- Weight Units:kg,lb,st:lb
- Capacity:150kg/330lb/23st:8lb
- Graduation: 0.1kg/0.2lb/0.2lb(1lb,≥20st)
- Overload indication: “ — — — — ”
- Low battery indication “Lo”
- Power supply: 1 x CR2032 battery

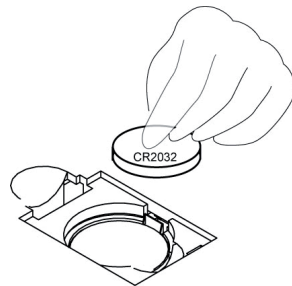
LCD Full Display

TO CHANGE BATTERY

- Ensure +/- terminals are correct way around.
- Use proper tool to help release the battery from the compartment if necessary.

Please remove the insulating strip before use.

FUNCTION KEYS

OPERATION INSTRUCTIONS

- 1) Press UNIT key to switch on and select unit.
- 2) Place the scale on the solid and flat ground ,let the scale turn off automatically.
- 3) User stand on the platform of scale directly, it will flicker and lock on the LCD after the weight steady, then automatically turn off after 10 seconds.
- 4) If without any operation , it'll turn off automatically
- 5) When you don't use it ,please balanced lay it, then do not put any item on the scale.

Kindly reminder:

** Don't use scale on the carpet*

** In order to ensure weighing accurate, when you move the scale, please first turn on(or tap on) and wait for turn off, then , you can stand on the platform again and measure your weight . Otherwise ,it will add scale own weight.*

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment. For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.*

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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