GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 - fax 199 400 403
Export: tel. +39 02 953854209/221/225 fax +39 08 95380056
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

POLTRONA ELETTRICA LUXOR LUXOR ELECTRONIC CHAIR FAUTEUIL ÉLECTRIQUE LUXOR SILLÓN ELECTRÓNICO LUXOR

Manuale utente - User manual - Notice d'utilisation - Manual del usuario

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




Chapter 1: INTRODUCTION

The design of the Luxor chair is smooth, simple and graceful, so it is suitable for any doctor's office.

- 1) This product has a better stability (it is more stable when adjusting the chair up and down), and it fits broad patients. (The maximum load is 200 kg).
- 2) The iron frame is packaged by plastic enclosures form. Its cover reaches 99% of chair and allows a better security.
- 3) Surface adopts the PU leather. This fabric is more comfortable and breathable compared to conventional materials.
- 4) The surface heating function is optional.

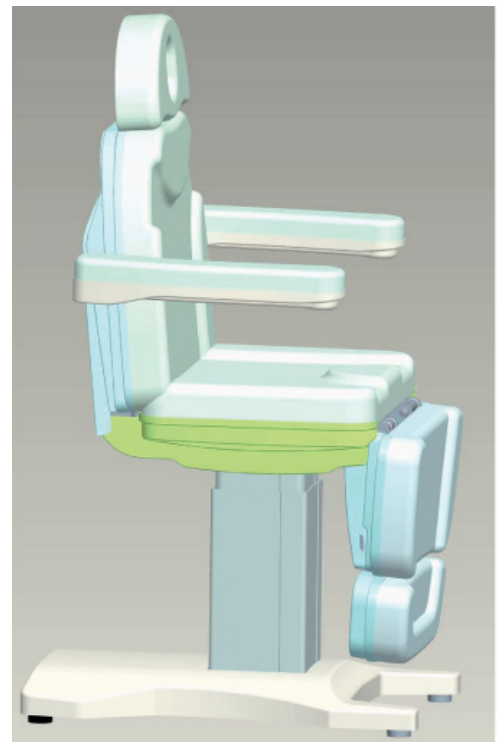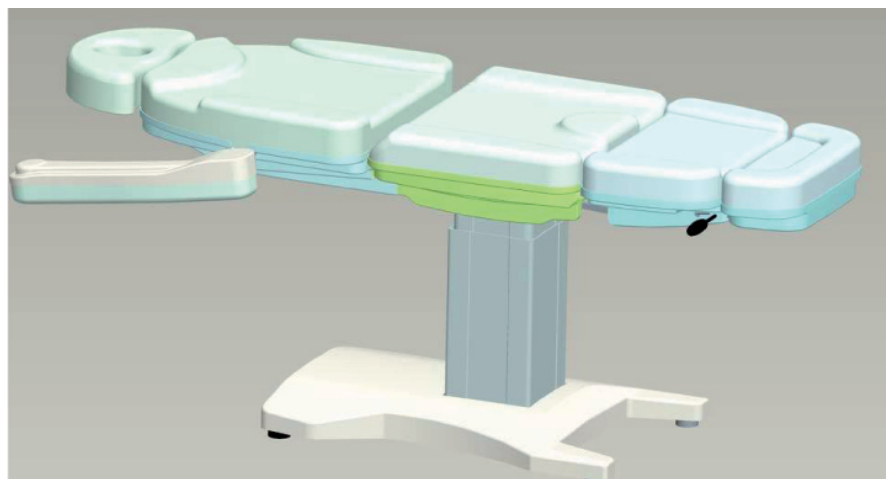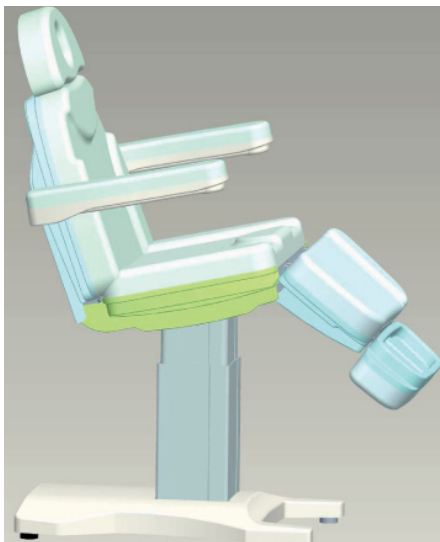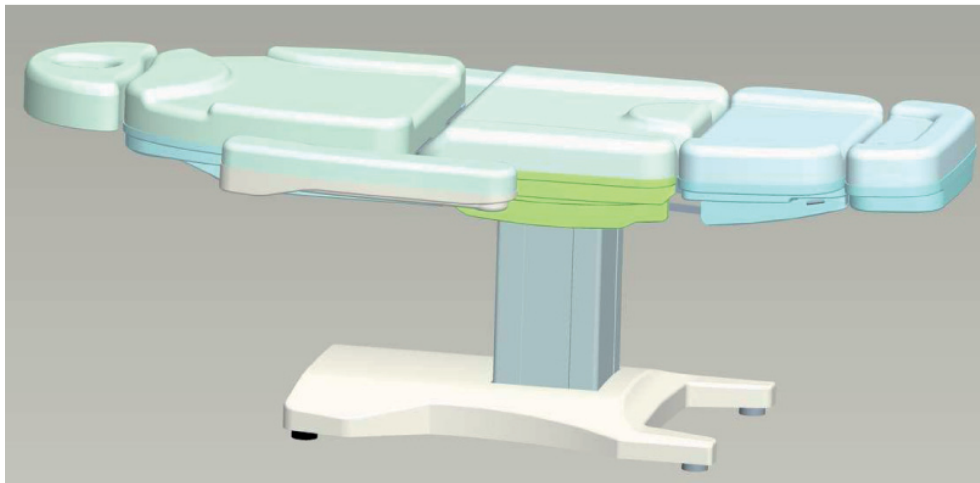
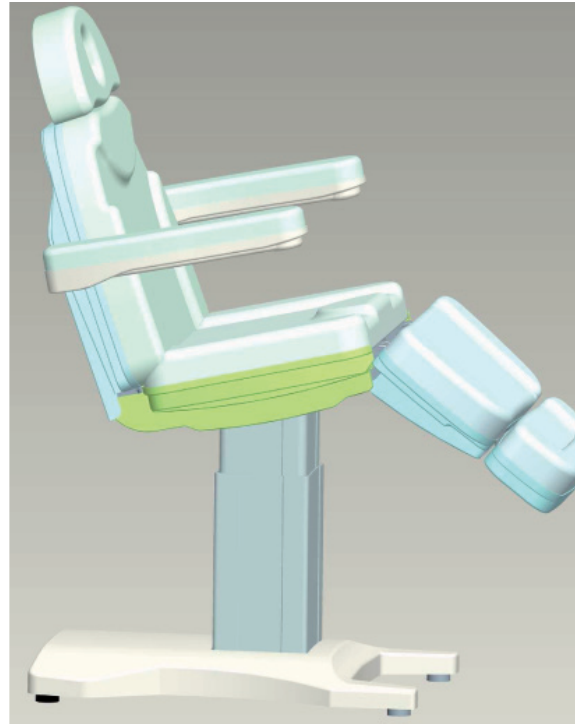
Chapter 2: ADJUSTMENT ILLUSTRATIONS


Chapter 3: TECHNICAL SPECIFICATIONS

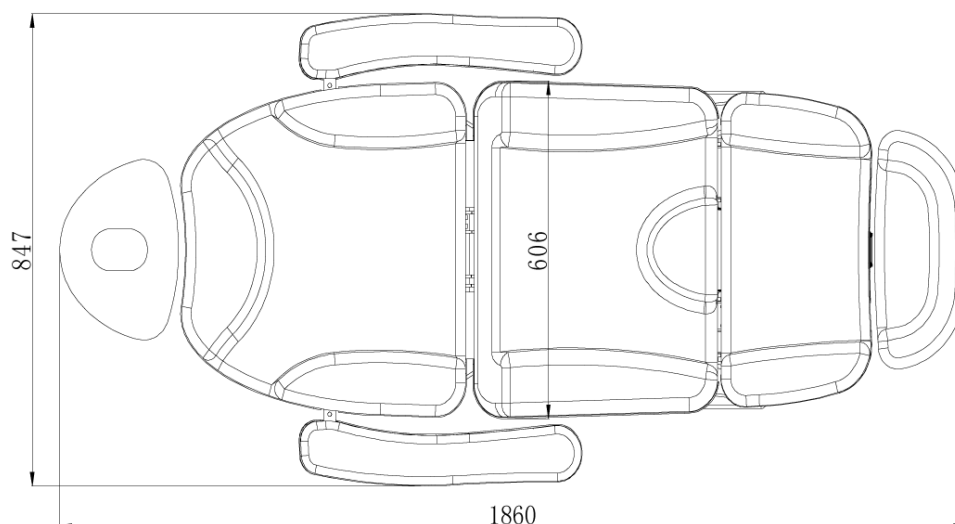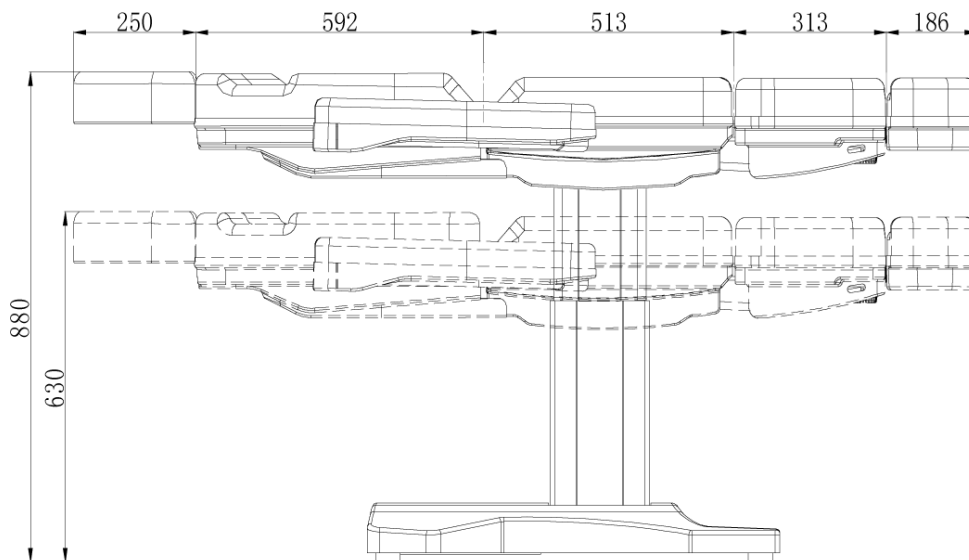1. Product size: 1860 x 880 x 630 (880) mm
2. Packing size: 1460 x 720 x 660 mm
3. Net weight: 85 kg, Gross weight: 95 kg
4. Height: Minimum 630 mm, Maximum 880 mm
5. Back adjustment angle: 0-85°
6. Foot adjustment angle: 0-90°
7. Cushion adjustment angle: 0-15°
8. Bed load limits of up and down: 200Kg
9. Head adjustment angle: $\pm 35^\circ$, 28-88mm head adjust-ment height, 80mm head adjustment length.
10. Leg adjustment length: 150 mm


Chapter 4: INSTALLATION AND OPERATION

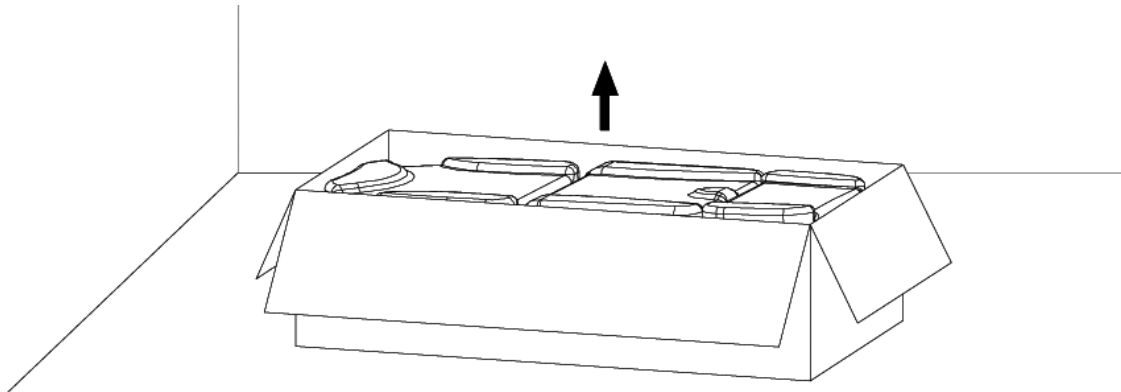1. Open the package. Take out product from carton.

N. 2

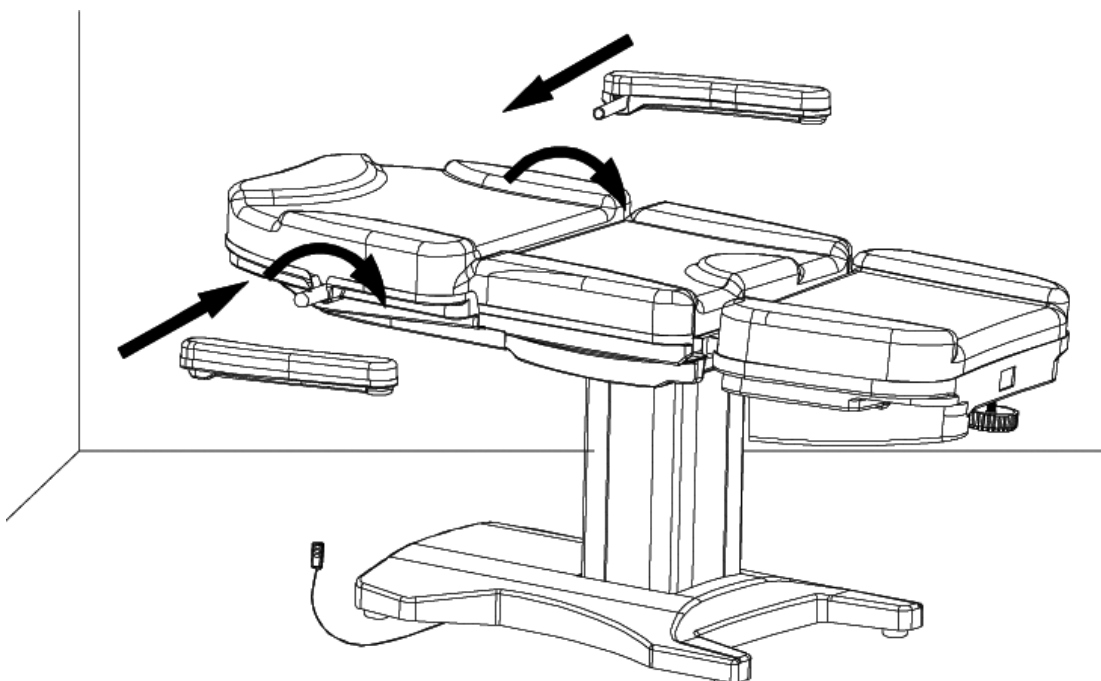
2. Install and test the fixture of armrest.

N. 3


- The shaft armrests on both sides of the back should be rotated as the picture shows, until they can not rotate.
- The simulated leather cover of armrests should be inserted upward to the armrests shaft, the two holes on the iron pipe of the armrest should be against the screws of armrests shaft.
- The stainless steel screws in the accessory should be fixed from the bottom up in the shaft on the armrests.

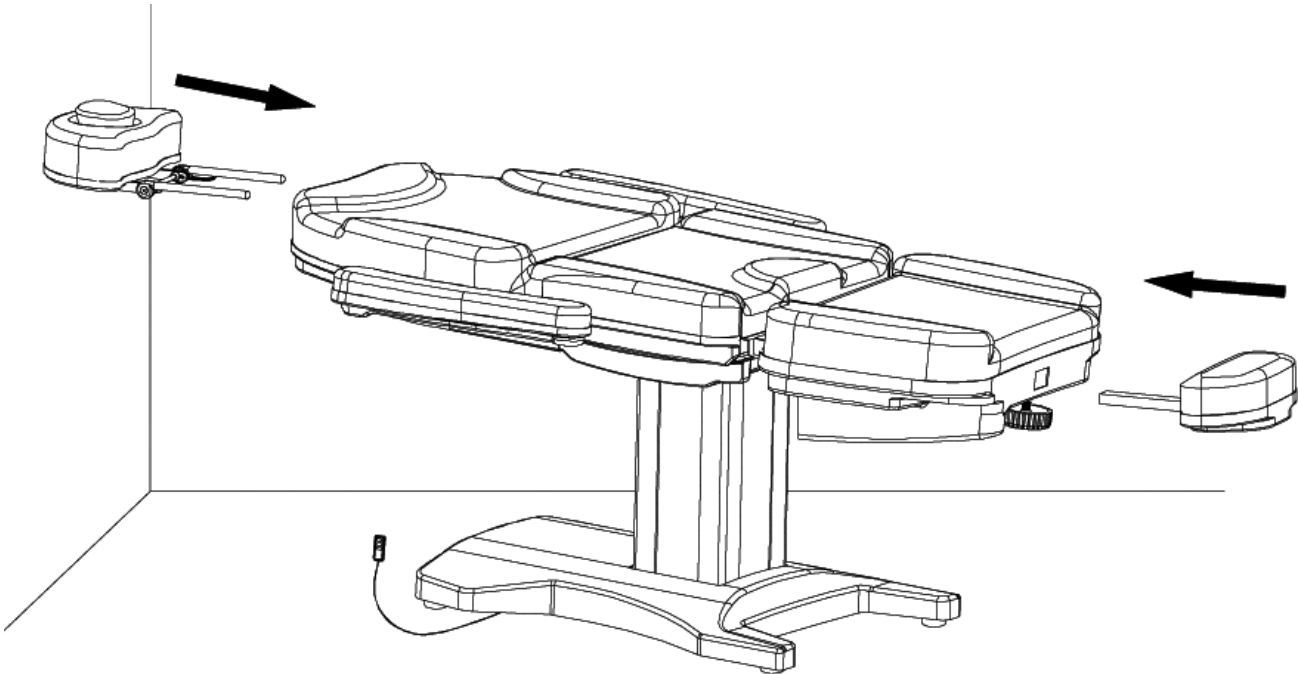3. Pillows and moving foot pad should be inserted into the back part and foot part respectively and fixed.

N. 4

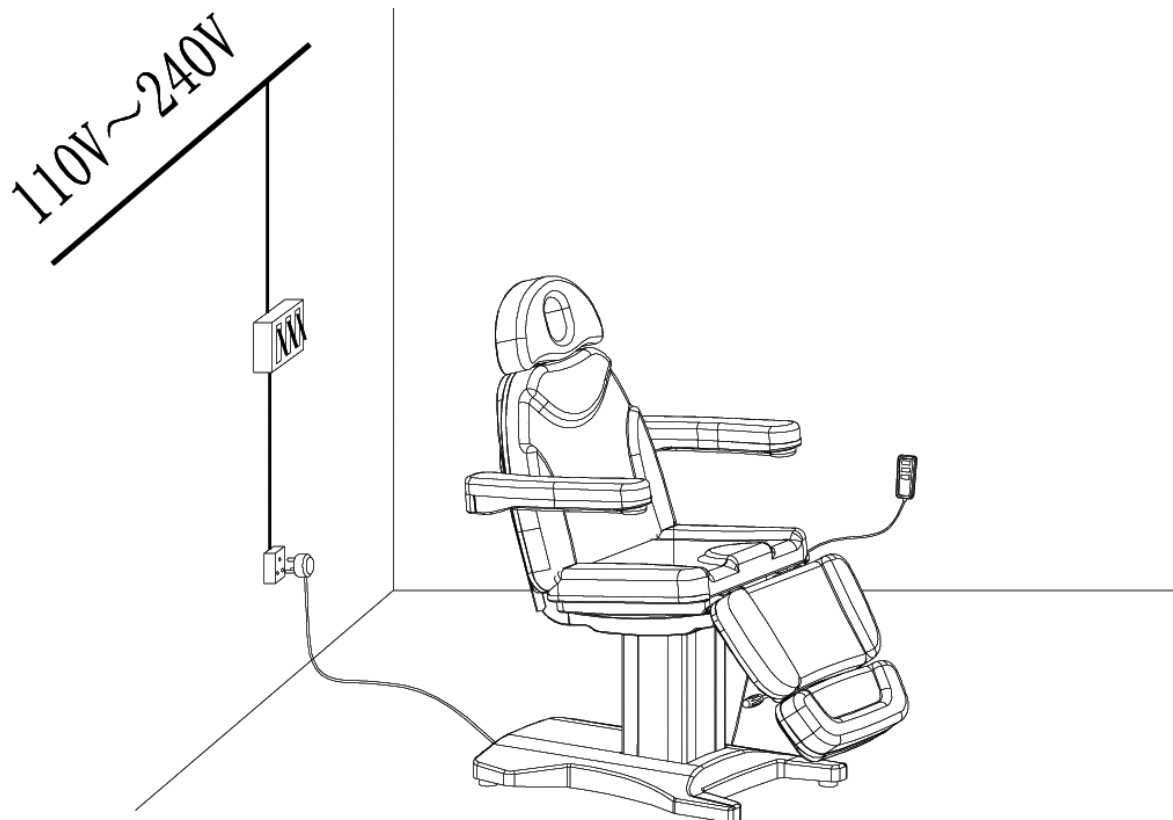4. The power cord plug of chair should be inserted into the junction box.
The installation of junction box must be through the air switch.

N. 5

Chapter 5: OPERATION

- 1) When increasing the angle of back part or decreasing the angle of leg part, if fingers are on the position "1" (as shown by picture 6) the phenomenon of pinching finger will easily happen.
- 2) When adjusting the height of the bed press the bed adjust bottom till the required height.

Caution:

- Do not put your hand, foot, head in a position that could block the adjustment of chair.
- Pay attention not to introduce body parts into moving and/or rotating components.
- The chair must be used under good stability conditions. The chair must rest on a flat surface.
- Keep chair still while moving patient on and off of chair because it could fall over.
- Do not allow patient to climb on chair from back side.

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment. For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.*

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA. During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included in the warranty. The repair or replacement performed during the warranty period shall not extend the warranty. The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use. GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc. The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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