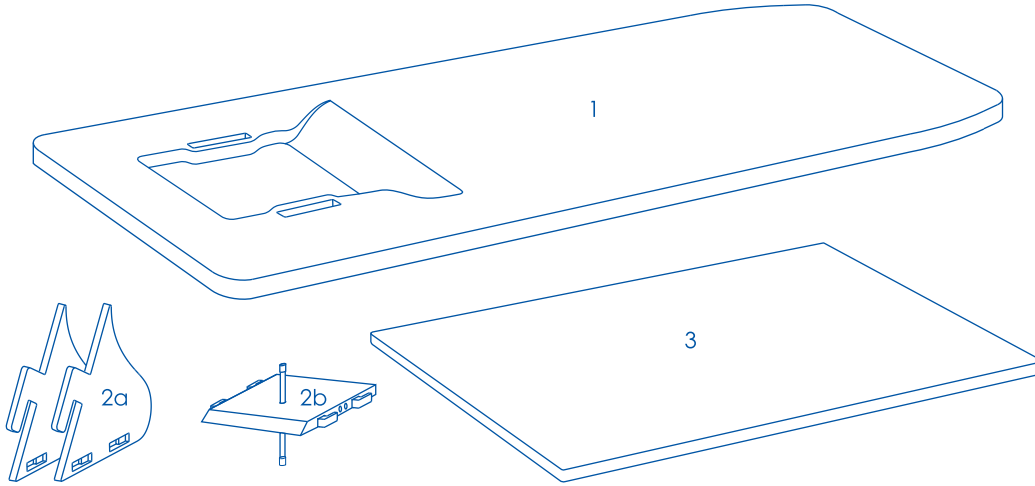
# My Perfect Colon<sup>®</sup>

Natural Hydro-Therapy at Home

## Tavola Board

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## MY PERFECT COLON BOARD

### **EN** INSTRUCTIONS: INSTALLATION AND USER GUIDE

- 1) Mount the cover back side (2b) into the lateral walls (2a): insert the "L" joints of the cover back side (2b) into the housing of the lateral side matched with the same number of holes (1 or 2) and push until it stops (fig. 1b).
- 2) Mount the "splash" cover on the board: insert the "L" joints into the two rectangular holes located near the board's large central hole and push until it stops (fig.2).
- 3) Place the side of the board with the hole on the toilet, making sure that the hole of the board is completely free and not blocked by the w.c. or by the edges of the toilet, so that it does not get dirty during cleansing (fig.3).
- 4) Put the other side of the My Perfect Colon board on a chair (with a height similar to that of the w.c.), that you have previously positioned in front of, or next to, the w.c. (fig.3).
  - For greater stability, the support (chair, bidet, stool, etc.) should be no further than 40 cm away from the w.c., so that the board may be leaned against the support at least as far as the blue line marked on the lateral edge of the board (fig.3).
  - To help the drainage of water and fecal residue, we advise putting the board in a position slightly inclined towards the drainage hole. Make sure that you never lean the board in the opposite direction.
- 5) Connect the modeling tube with check valve of My Perfect Colon device (not included) to the back joint of the tube inserted in the cover (fig.4).
- 6) Screw a rectal pipe (not included) to the tube inserted in the cover (fig.4).
- 7) Connect the My Perfect Colon device to the tap and try turning it on slightly to verify that all connections are correct.
- 8) Having regulated the temperature, turn off the water flow through the My Perfect Colon flow regulator.
- 9) Place the comfort pad on the board (leaving free the drainage area) and then, holding the flow regulator in your hand, lie down on the board, positioning your buttocks near

the cover (fig.5). Your feet can then be placed on the end of the board or against the wall near the w.c. For a more comfortable cleansing, we suggest using a pillow under the head.

10) Insert the rectal pipe into the colon to about 5-6 cm, regulating the position of the tube in the cover.

11) Once in position, turn on the water flow through the flow regulator and start the cleansing.

- For the colon cleansing follow carefully the indications contained into the manual of the device My Perfect Colon.
- To improve the effectiveness of the cleansing and to facilitate the expulsion of fecal residue, we suggest massaging the abdomen with circular movements.
- Water and fecal residue will be eliminated and flushed into the w.c. through the drainage hole.

12) Once the cleansing is finished, turn off the water flow through the flow regulator. Wash the board before putting it back.

To use My Perfect Colon correctly, read the manual contained in the packaging carefully.

My Perfect Colon is a CE medical device.

### **WARNING:**

- Do not use the board if it shows signs of cracks or breakages.
- For people weighing more than 100 kg, we recommend using an adding support (chair, stool, etc.) under the central part of the board.
- Maximum weight supported 120 kg.
- Before sitting and lying on the board, check that the board is stable.
- Do not stand on the board and do not perform any sudden movements which might compromise its stability.
- If the board is to be used by old people, or those of restricted mobility, we recommend that the help of another person be enlisted.
- Do not use the board for purposes and uses other than those set forth in this manual.