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
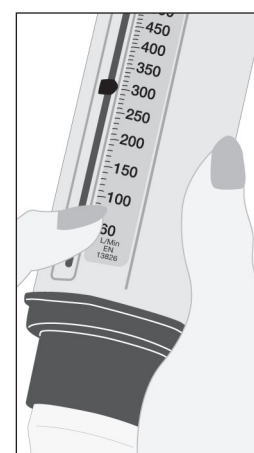
1 en Insert the mouthpiece into the meter, if not already fitted. Ensure the pointer is set at Zero (L/MIN position).


2 en Hold the Peak Flow Meter so that your fingers are clear of the scale and slot. Do not obstruct the holes at the end of the Peak Flow Meter.

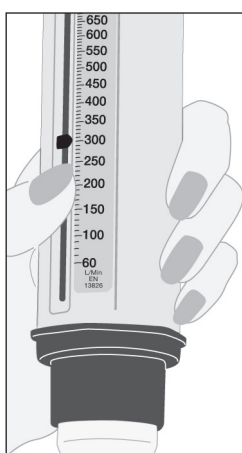
3 en Stand up if possible. Take a deep breath, hold the Peak Flow Meter in the mouth and hold horizontally, closing the lips around the mouthpiece, then blow as hard and as fast as you can.

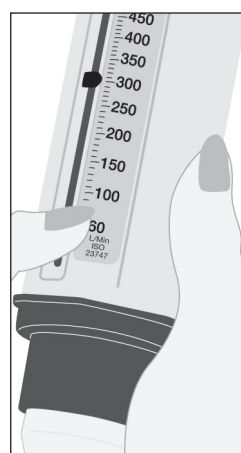
4 en Note the number on the scale indicated by the pointer.

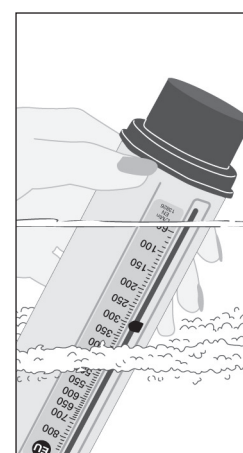5 en Return the pointer to zero (L/MIN position) and repeat the procedure twice more to obtain three readings. Mark the highest of the three readings on the chart as shown overleaf.

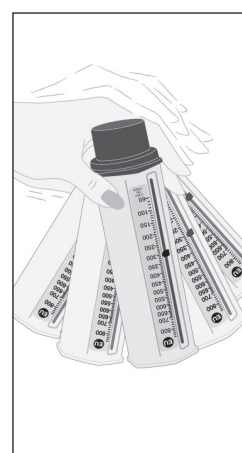en CLEANING YOUR PEAK FLOW METER Immerse Peak Flow Meter in warm (but not hot) mild detergent solution for 2-3 minutes (maximum 5 minutes). Agitate the meter to ensure thorough cleaning.

Rinse in clean warm water and shake gently to remove any excess water. Allow to dry thoroughly before using again.

Standard Peak Flow Meter en 1 Standard Peak Flow Meter complete with detachable Universal Mouthpiece and Instructions/Recording Chart

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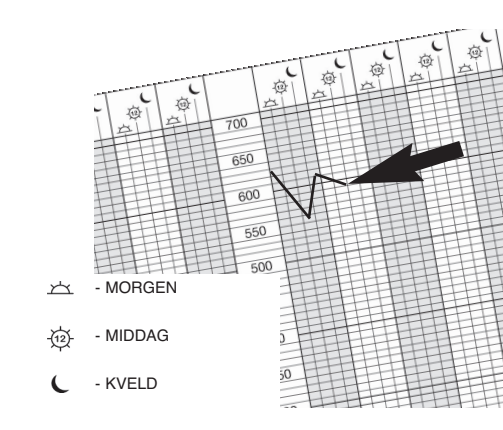
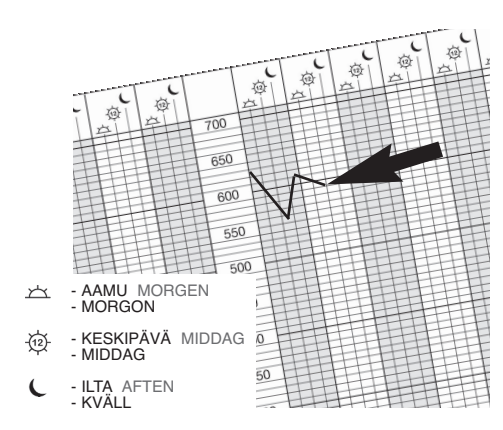
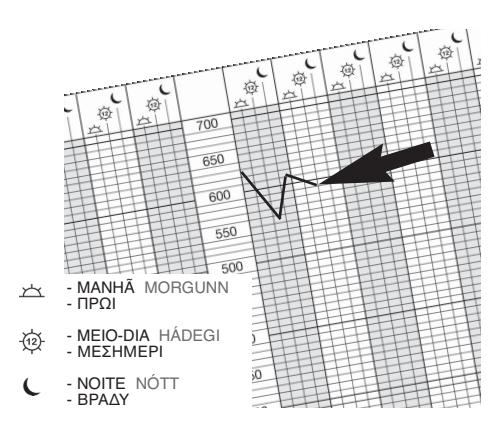
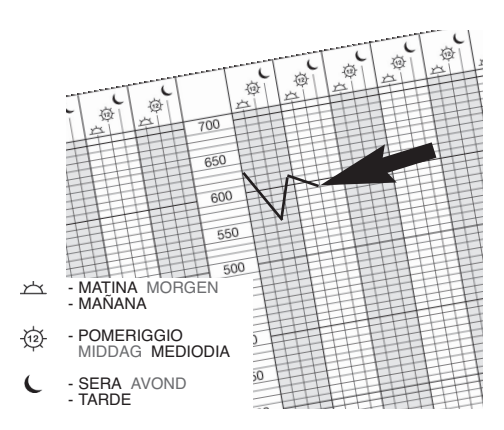
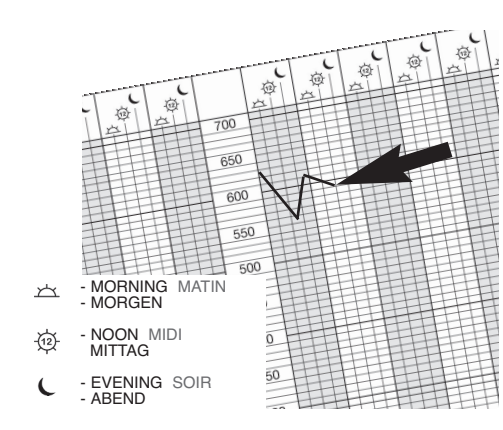
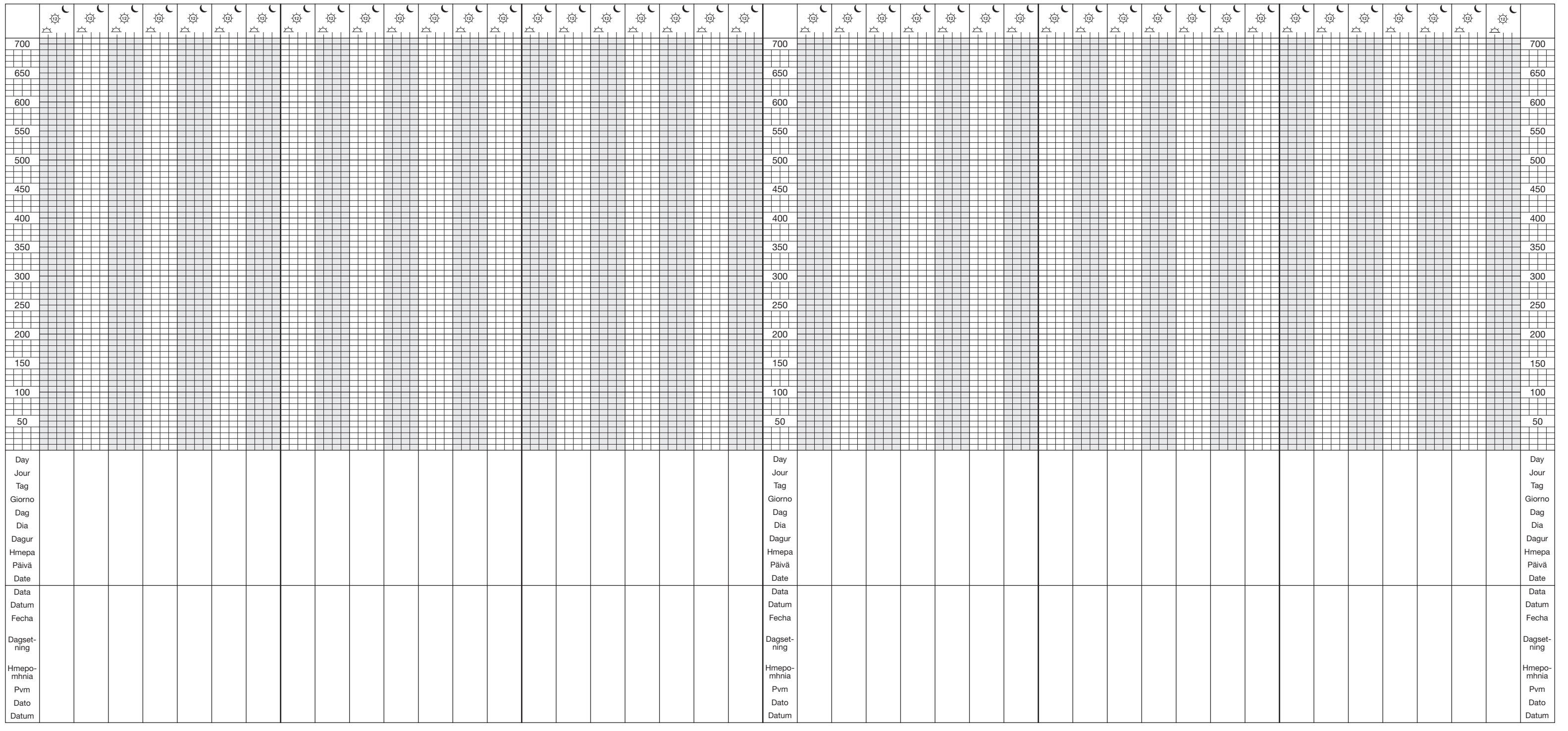
en HOW TO RECORD YOUR PEAK FLOW

It is important to keep a record of your peak flow. Readings should be taken twice a day, first thing in the morning, and at bed time, or as directed by your physician. Use the chart provided to record the highest reading obtained each time. Mark your reading on the chart with a dot corresponding to the highest reading obtained for each of the morning and bed time readings every day. By joining the dots (see diagram) you can see how your peak flow varies from day to day. You should also measure and note your peak flow if you suddenly become wheezy or breathless at any time of the day or night.

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