



# EASYRED

## MASCHERA MONOUSO AIR CUSHION

### MODELLI

|              |          |
|--------------|----------|
| <b>RN440</b> | MISURA 0 |
| <b>RN441</b> | MISURA 1 |
| <b>RN442</b> | MISURA 2 |
| <b>RN443</b> | MISURA 3 |
| <b>RN444</b> | MISURA 4 |
| <b>RN445</b> | MISURA 5 |

### AVVERTENZE

Conservare in un luogo asciutto, lontano da fonti di calore e luce. Il paziente deve essere monitorato costantemente ogni volta che l'apparecchio è in uso. L'utilizzo è destinato ad un unico paziente. Il riutilizzo può provocare infezioni. Non mettere a bagno, sciacquare, lavare o sterilizzare questo prodotto. Queste procedure potrebbero lasciare residui dannosi sulla maschera. Distruggere dopo l'uso.

### ISTRUZIONI PER L'USO

Prima dell'uso verificare il passaggio dell'aria. Collegare al circuito di respirazione o all'apparecchio di rianimazione in modo appropriato la maschera. Assicurarsi che tutte le connessioni siano compatibili e siano ben fissate.

**Prodotto da:**

BESMED HEALTH BUSINESS Corp.

NO.2, Lane 106, Wu-Kong 3rd Road, Wu kong Industrial Park, Taipei Hsien. Taiwan.

**Mandatario stabilito U.E.:**

Star Trade Srl. Via Sismondi,63 - 20133 Milano - Italy

**Distribuito da:**



**MORETTI**

Moretti S.p.A.

Via Bruxelles 3 - Melegnano

52022 Cavriglia (Arezzo)

Telefono +39 055 96 21 11

Fax +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)

[info@morettispa.com](mailto:info@morettispa.com)



# EASYRED

## CUSHIONED DISPOSABLE FACE MASK

### MODELS

|       |        |
|-------|--------|
| RN440 | SIZE 0 |
| RN441 | SIZE 1 |
| RN442 | SIZE 2 |
| RN443 | SIZE 3 |
| RN444 | SIZE 4 |
| RN445 | SIZE 5 |

### CAUTION

Store in clean dry conditions away from heat and light.  
patient should be monitored constantly whenever this device is in use.  
For single patient use. Re-use can result in cross-infection. Do not soak, rinse, wash or sterilize this product. These procedures may leave harmful residues on the mask. Destroy after use.

### DIRECTIONS FOR USE

Check for clear air flow passage prior to use.  
Connect to breathing circuit or resuscitation device with appropriate fitting. Make sure all connections are compatible and fit firmly.

**Manufactured by:**

BESMED HEALTH BUSINNESS Corp.  
NO.2, Lane 106, Wu-Kong 3rd Road, Wu kong Industrial Park, Taipei Hsien. Taiwan.

**Eu established mandatory:**

Star Trade Srl. Via Sismondi,63 - 20133 Milano - Italy

**Distributed by:**



**MORETTI®**

Moretti S.p.A.

Via Bruxelles 3 - Melegnano  
52022 Cavriglia (Arezzo)  
Telefono +39 055 96 21 11

Fax +39 055 96 21 200

[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)