**METRO MISURA NEONATI**  
**BABY MEASURE METER**  
**MÈTRE DE MESURE POUR NOUVEAU-NÉS**  
**MESSMETER FÜR SÄUGLINGE**  
**TALLÍMETRO PARA BEBÉS**  
**METRO PARA MEDIR RECÉM-NASCIDOS**  
**ΜΕΤΡΟΥ ΜΕΤΡΗΣΗΣ ΝΕΟΓΕΝΝΗΤΩΝ**  
**METRA DO MIERZENIA NOWORODKÓW**  
**ÚJSZÜLÖTT MÉRŐSZALAG**

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The baby measure meter GIMA is a precision device for measuring the height of infants in centimeters and assess the growth curve. The baby measure meter is intended for use in pediatric clinics, hospitals for children or for home visits by pediatricians. On the upper fixing bracket are present holes for hanging the mat to the wall when not in use. The mat can measure up to a maximum of 1 meter, with an accuracy of + / - 2 millimeter.

## USE

### How to make a measurement

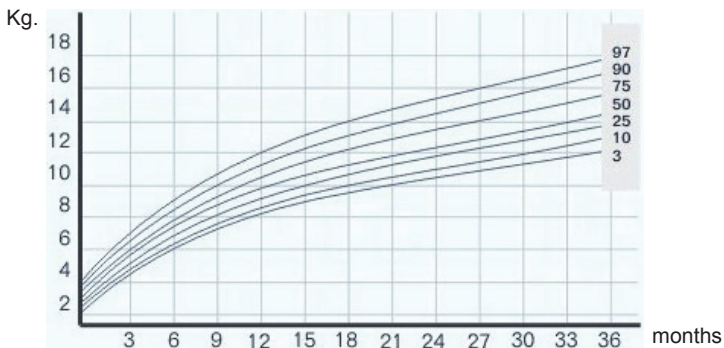
1. Place the baby lying on the graduated mat after placing stops in ABS correctly at 90 ° vertical to the scale. The baby's head must be resting on stop in ABS holding firmly with the left hand the infant.
2. With the right hand approach to baby's feet the stop placed at the end of the graduated scale.
3. Be careful to keep the legs of the baby very straight. It is recommended to gently press with the left hand the knees of the baby towards the mat.
4. When the final stop is positioned to the baby's soles at a right angle, read the height indicated on the bottom of the graduated mat.

By following the above instructions you can achieve a high level of accuracy.


## INTERPRETATION

Measure the height of the baby is useful to determine the proper growth curve by calculating percentiles, in combination with the weight. The "weight and height" of infants can be compared with reference curves called Tanner-Whitehouse. These curves identify the lines (percentiles) that define the range considered normal and acceptable according to age.

**Weight: males from 0 to 3 years**

**Weight: females from 0 to 3 years**

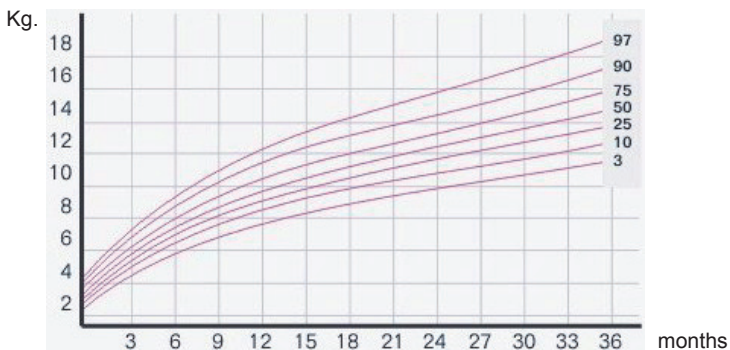It is desirable that the growth curve is not over and above or below the average (percentile 50). A percentile of 50 (in the midline) in height means that 50% of children of his age are taller and the other 50% are lower. The utility of these curves is to highlight any differences that might be assumed (and therefore suggest to make a medical exam) obesity, lack of bone growth or short stature.

### CLEANING

Clean the mat with a mild detergent and a soft cloth.

### SAFETY TIPS

- *Never leave the baby lying on the mat without the supervision of an adult.*
- *Take measurements only on safe contact surfaces (from which the baby cannot fall.)*
- *Clean and disinfect the graduated mat regularly*

### GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included.

All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

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