AVVERTENZE

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WARNINGS

Do not immerse the device in water
 Switch off the unit before removing the plug
 Remove the plug before opening the unit for any reasons (bulb replacing)
 Do not introduce foreign bodies into the small holes of the device
 Do not open or attempt to adjust the device without GIMA authorisation such action invalidates the guarantee

To polish the device use damp towelling.
 To disinfection use damp towelling specificity and rub over all device

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INSTRUCTION FOR USE

Rimsa Lamps are sold assembled and packed in cartoons box.

After taking the lamp out of the packing, proceed as follows

A) mod. GIMANORD with caster base:

- 1-insert the pivot of the lamp into the hole upper the caster base.
- 2- screw the black screw to avoid the lamp structure comes out the tube
- 3-insert the electric plug, which is attached at the end of electric cable, in the ground connection
- 4-switch-on the lamp by turning the dimming.

**B) mod. GIMANORD wall type (wall clamp S/12)**

- 1-fixing the clamp S/12 to the wall with no. 3 expansion screws.
- 2-insert the lamp into the hole of the wall clamp S/12.
- 3-see points 2, 3 and 4 above mentioned

Before any maintenance, such as substitution of bulbs, log out the lamp and remove the cable from the ground connection.

Bulb replacement

- 1-remove the 4 screws positioned on sides of the aluminium reflector
- 2-remove the plexiglass protection
- 3-substitute fluorescent circline 22W-230V G10q
- 4-put the plexiglass to original position and screw the 4 screws on sides of reflector.

TECHNICAL SPECIFICATION

Fluorescent circline 22 W / 230V G10q

Power supply 230 V

Nominal Frequency: 50Hz

Classification: Class I

Level protection against liquid entrance: IP20

Made in Italy

GIMA s.p.a. Via Trento,10 - Gorgonzola (MI) Italia

Having seen the compliance statement of the manufacturer, GIMA s.p.a. declares that this device has been produced according to norm EN 60598-1 and conforms to European Directives 73/23 CEE and EMC 89/336 CEE.

SIMBOLS EXPLANATION

V =nominal supply voltage

Hz =supply frequency

VA = lamp consumption

 = earth

~ =supply voltage type